# Auto-reader

for Rapid, Super Rapid and  
Ultra Rapid Biological Indicators

*Innovation in technologies for Infection Control*

Warning: this product contains dry natural rubber

MiniBio

**TERRAGENE**



Rev.8 | May 2020

# Index

<b>Composition</b> .....	<b>4</b>
Product description.....	4
Indications for Use.....	5
References.....	5
<b>Safety information</b> .....	<b>5</b>
Symbols.....	5
Operating conditions.....	6
Power supply specifications.....	6
Environment operating conditions.....	7
Device Safety Compliance.....	7
Product Compatibility and Incubation programs.....	7
Bionova® MiniBio Incubation Programs.....	7
Bionova® MiniBio Auto-reader compatible indicators.....	7
<b>Serial/Batch Number</b> .....	<b>7</b>
<b>Instructions for use</b> .....	<b>8</b>
Start-up.....	8
Remaining incubation time.....	9
Positive Control.....	9
Interpretation of results .....	10
Disposal.....	10
Audible alarm .....	10
Cancellation of a reading.....	10
Temperature monitoring.....	10
Thermal paper specifications.....	11
Replacement of the paper roll .....	11
Reprinting of results .....	11
Setup Mode .....	11
Time setting mode.....	12
Cleaning and maintenance.....	12
<b>Additional features</b> .....	<b>12</b>
Firmware update.....	12
Bionova® Traceability Software.....	13
<b>Troubleshooting</b> .....	<b>14</b>
<b>Warranty</b> .....	<b>15</b>
Limitation of liability.....	15
Technical assistance.....	15

## Composition

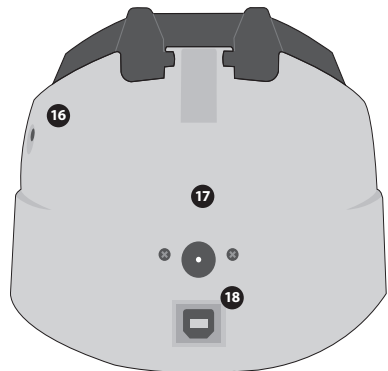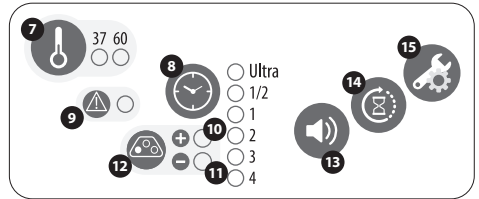### Product description

Bionova® MiniBio Auto-reader has been designed for the incubation and automatic readout of Terragene® Rapid, Super Rapid and Ultra Rapid Self-contained biological indicator(SCBIs) appropriate for a wide range of sterilization control applications. Bionova® MiniBio allows two different incubation temperatures, 37°C or 60°C. Microorganisms contained within the SCBIs vary depending on the sterilization process for which they have been designed, therefore, incubation temperatures and incubation times differ depending on the SCBI being used. Bionova® MiniBio allows independent time settings selection for each one of the three incubation positions.

SCBIs for Ethylene Oxide (EO), should be incubated at 37 °C, while those used for the monitoring of the rest of the sterilization processes should be incubated at 60 °C.

Simultaneous incubation of SCBIs for monitoring EO processes along with SCBIs for monitoring other sterilization processes is not possible.

Please check the **Product Compatibility and Incubation programs** section for more information about Incubation Programs and new available Indicators for your Bionova® MiniBio.

Bionova® MiniBio Auto-reader allows easy and rapid detection of positive and negative SCBIs using advanced fluorescence techniques. A positive result can also be evidenced by culture medium color change when performing extended incubations. Read the instructions for use of each SCBI for more information. The option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital.

Bionova® MiniBio Auto-reader also provides a printed ticket each time an incubation is completed for recording the results. This allows for easy result management, for documentation compliance and safe-keeping.

## Safety information

### Indications for Use

Terragene® Bionova® Reader Incubators (Mini-Bio) incubate at 60°C and 37°C and read the Terragene® Bionova® SCBI for fluorescent results at the times prescribed in the User Manuals.

### References

- 1 Protective Cover.
- 2 Incubation Area.
- 3 Control Panel.
- 4 Ampoule Crusher.
- 5 Cavity for Paper.
- 6 Thermal Printer.
- 7 Incubation Temperature Selection Button.
- 8 Incubation Time Selection Buttons / Time Left.
- 9 Blue Attention Light.
- 10 Red Light / Positive Indicator.
- 11 Green Light / Negative Indicator.
- 12 Incubation Position.
- 13 Alarm Cancellation Button.
- 14 Reprint of Results.
- 15 Configuration / Paper Traction Button.
- 16 Hole for External Temperature Control.
- 17 Input for Power Supply Plug (12Volts DC).
- 18 USB Port.

### Safety information

#### Symbols

-  30% - 80% Operating relative humidity.
-  10 °C - 30 °C Operating temperature.
-  Caution, Warning, Attention - Refer to the instructions for use.
-  Caution: Hot surface.
-  Important.
-  Direct current.
-  For indoor use only.
-  Keep away from sunlight.
-  Batch code.
-  Manufacturer.
-  Separate collection for waste of electrical and electronic equipment.

#### To avoid risks and/or damaging the device:

- For indoor use only.
- Do not place the incubator in a room exposed to direct sunlight or to high luminous intensity lamps.
- Do not place the incubator near devices that emit strong electromagnetic fields.
- Do not use the incubator on leaned surfaces or on surfaces that are subject to shocks, vibrations, temperature or high relative humidity.
- Disconnect the power cord before cleaning.
- Do not use abrasive, corrosive cleaners or disinfectants.
- Do not immerse into any liquid . Do not pour any liquid inside.
- Make sure the incubator is connected to an appropriate electrical mains outlet socket.
- Use only the included power supply (AC power)

## Safety information

adapter), power supply's AC plug, power supply cords, and USB cable. Verify that all the included elements are in good condition on daily basis. If any of them is damaged, discontinue their use. The use of cables, adapters, cords, and/or power supplies different from the ones included may cause fires, electrical shocks, or even physical injuries.

- Do not plug any devices into the Incubator's USB port other than a personal computer (PC). The PC has to be compliant with IEC60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its USB ports. Ask a qualified technician to verify device compatibility. Attaching any other device to the USB port may damage the incubator and may not be safe for the user.

- Do not attempt to repair the incubator by yourself, that could lead to major and irreversible damages to the device. In case of device malfunction, contact your local distributor for further assistance.

### To reduce the risk of using incompletely sterilized loads:

- Please read, understand, and follow the Instructions for Use of each SCBI before its incubation.

- Do not remove the SCBI before the incubator reports the final readout result. Check a result ticket is printed.

- Check that culture medium completely wets the spore carrier.

### To avoid the risk of injury, related to glass fragments produced when crushing the glass ampoule inside the SCBI tube:

- Cool the SCBI during the indicated time before crushing the ampoule.

- Do not handle the SCBI excessively since this might cause the glass ampoule to burst.

- Wear safety gloves and glasses when removing the SCBI from the sterilizer, pressing the SCBI's cap, and crushing the SCBI's ampoule.

- Do not use your finger for crushing SCBI. Use the Ampoule Crusher instead.

### To avoid a potentially hazardous situation:

- Avoid contact with the hot metal block inside each incubation position.

- Do not insert your fingers, or any other element, inside the incubation positions.

- Place only compatible indicators inside the incubation positions.

### To avoid SCBIs from absorbing fluorescent particles:

- Avoid direct contact between the SCBIs and Chemical Indicators or Tapes before the SCBI incubation.

### Important

Do not use this product in a manner not specified by Terragene® S.A., otherwise the protection provided by the product might be affected.

Only Terragene® S.A. authorized personnel can access or service the internal components of the Incubator. Parts or components inside the Incubator should not be manipulated by the user.

## Operating conditions

### Power supply specifications

Input parameters	Operating conditions	Units
Voltage range	(100-240)	AC Volts
Frequency	50/60	Hertz
*Current	0,5	Amperes
Output parameters	Values	Units
Voltage	12	DC Volts
**Current	2	Amperes

\* 1 Ampere in USA and Canada.

\*\* 3 Amperes in USA and Canada.

Terragene® S.A. recommends the use of UPS instead of voltage stabilizers, since they fulfill two functions: to stabilize and maintain the energy during a power outage.

## Safety information

### Environment operating conditions

Environmental Conditions	Operating Conditions	Units
Altitude	3500 (máx.)	Meters
Operation temperature	10-30	Celsius
Relative Humidity	30-80	%
Installation/overvoltage	category II	
Degree of contamination	2	
Storage temperature	10-30	Celsius
Voltage	12	DC Volts

### Device Safety Compliance

Bionova® MiniBio Auto-reader Incubator complies with the following standards and directives:

<b>Electrical Safety.</b>	IEC 61010-1 IEC 61010-2-010 Low Voltage Directive 2014/35/EU.
<b>Electromagnetic Compatibility (EMC).</b>	EN 61326-1 EMC Directive 2014/30/EU
<b>European Commission.</b>	RoHS Directive 2011/65/EU. WEEE Directive 2012/19/EU.

Bionova® MiniBio Auto-reader Incubator do not represent photobiological risk and do not generate dangerous optical radiation in any of its normal operation conditions as per the requirements of IEC 62471 Standard.

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

### Product Compatibility and Incubation programs

#### Bionova® MiniBio Incubation Programs

The following incubation programs are available for your Bionova® MiniBio. Verify which incubation programs are available for your country checking the *Indications for Use* section.

Temperature selection	Time selection	Incubation Program
<b>60 °C</b>	20 min ( <i>Ultra</i> )	20 min. at 60°C
	30 min	30 min. at 60°C
	1 hs.	1 hour at 60°C
	2 hs.	2 hours at 60°C
	3 hs.	3 hours at 60°C
<b>37 °C</b>	4 hs.	4 hours at 37°C

To select the appropriate incubation program for the Indicator you are planning to use, follow each Indicator's Instructions for use.

Select the appropriate incubation program before the incubation of any Indicator.

#### Bionova® MiniBio Auto-reader compatible indicators

We develop new Indicators for extending the features of our incubators regularly. Please visit [www.terragine.com.ar/compatible](http://www.terragine.com.ar/compatible) or scan the QR code below to check the latest compatible Indicators developed for your incubator.

### Serial/Batch Number

For easy identification, each Bionova® MiniBio Auto-reader Incubator has a distinctive serial and batch numbers. The number can be found printed on two labels, one placed at the bottom of the Incubator, and one on its packaging. Please, register your serial and batch numbers in this manual for future reference:

**Serial number:** \_\_\_\_\_

**Batch number:** \_\_\_\_\_

# Instructions for use

## Instructions for use

### Start-up

1. Place the Bionova® MiniBio Auto-reader Incubator on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Incubator in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Incubator to the closest wall. Do not move the Incubator periodically or during its use. Connect the Incubator to a secure and stable electrical mains outlet socket.

Do not wet or heat the device. If liquid is spilled on the Incubator, disconnect it and follow the instructions on the **Cleaning and maintenance** section.

2. Power on your Bionova® MiniBio by connecting the power supply's AC plug to mains and then connect the plug at the other end of power supply to the rear of the Incubator. The last incubation program used (time and temperature combination) will be selected by default.

Before power on, verify that all incubation positions are empty.

If the Blue Attention light starts blinking after powering on the Incubator, check that the printer's door is tightly closed and that the printer has paper, if not follow the instructions on the **Replacement of the paper roll** section.

3. Set the incubator internal clock to correspond to your local time. See configuration methods in the **Time setting mode** section.

All devices are manufactured with the following settings:  
- Time zone: UTC +0:00  
- Printing language: English.

If you want to change the time zone, or the printing language refer to **Modification of time zone**, or **Modification of printing language** on the **Set-up mode** section.

4- Check that required incubation temperature is correct. To change it, press button  for 3 seconds. Lights corresponding to 37 °C and 60 °C will start blinking, indicating that the

temperature program must be selected. Press the button  to select the incubation temperature program, 37 °C or 60 °C. After 3 seconds, temperature modification will be accepted.

For 37 °C, Bionova® MiniBio Auto-reader will automatically select 4 hs. final readout for all positions.

For 60 °C the incubator allows choosing between **Ultra** (20 minutes), 30 minutes, 1 hour, 2 hours or 3 hours final readout. It is possible to select an incubation program for each of the positions depending on the final reading characteristics of the indicators to be used.

Check that the time setting is appropriate for the SCBIs to be incubated in each position.

To change the time program, press the button  of the correspondent position as many times as necessary to choose the right setting.

5- Wait for the temperature of the machine to be stable. Once is stable, the indicator of the desired temperature will remain steady.

Do not place any biological indicator in read positions until the incubator does not confirm temperature stability.

The incubator will keep running the incubation program during all reading process. The system does not allow the incubation program to be modified once the reading is started, except it has been completed.

6- Reading can be initiated when the temperature blue light remains stable indicating that incubation temperature was met. Then an automatic testing of the status of the incubation positions of the biological indicators will be performed. The indicating lights of each position will be turned on to show the result: green light will show a satisfactory result while red light will indicate error.

When the device detects an error in any position, it will print a ticket indicating date, time and positions with error. To guarantee the reliability of the results, these positions will be deactivated avoiding their use. To make this clear, red light in that position will remain blinking indefinitely to indicate that the position has been disabled.

For the correct performance of the automatic check, it should be done with the protective cover closed.

Read SCBI's instructions for use. Before placing the vials in the incubator, press the top to seal the tube. Crush the internal ampoule (using the

## Instructions for use

ampoule crusher within the box of SCBIs or the one located within the incubation area of the incubator) and make sure that the liquid has completely wet the spore carrier placed at the bottom of the biological indicator plastic tube.

Do not remove or change placement of SCBI once the reading process has begun. If this occurs, results may be invalidated.

Use a non-sterilized biological indicator as a positive control every time a processed indicator is incubated. For more information refer to the **Positive Control** section of this user guide.

7-Place the rapid SCBI in a reading position, close the protective cover and wait for the result. Reading will start automatically. Immediately after placing the SCBI in a position, Bionova® MiniBio Auto-reader will emit a sound alarm indicating that reading has successfully began and status position light will turn on and will blink until the result is informed.

**NOTE:** If the incubator does not run automatically, it might indicate that the incubator temperature is not steady yet (check the temperature indicating light to remain steady) or that the incubation position is disabled due to an error in the initial autotest.

8- When a positive result is detected, the red light and the audible alarm will turn on. This will indicate that the sterilization process to which the SCBI was subject, has failed. 30 seconds after removing the SCBI from the incubation position, the alarm and the red light will turn off automatically and the position will be available to start a new reading.

The negative SCBIs will be informed with a green light in the corresponding position, this will indicate that the sterilization process to which it was subject has been successful. The green light will remain on for up to 30 seconds after the SCBI is withdrawn from the reading position.

The incubator will inform results within 20 min. (60° C), 30 min. (60 °C), 1 h. (60 °C), 2 hs. (60 °C), 3 hs. (60 °C) or 4 hs. (37 °C), according to the program selected. These periods of incubation are the time-limits used by the equipment to inform the fluorescence final readout.

9- Every time a positive result is detected, it will be informed through the printing of the corresponding ticket and with an audible alarm. The alarm can be cancelled by pressing the button .

10- To register the tickets, take the paper and press up for the paper to be cut with the serrated edge of the printer.

**NOTE:** If the paper for ticket printing is finished, the attention blue light will flash rapidly. To replace the paper, follow the instructions in **Replacing the roll paper in the printer**. The incubator will save the last 3 results, for re-printing, follow the **Reprinting of the last results** instructions.

<p>BIONOVA MiniBio RAPID READOUT INCUBATOR SERIAL NUMBER: XXXX XXX TICKET #: 000004 PROGRAM: X h / XX °C DATE: DD/MM/AA START TIME: HH:MM READOUT TIME: HH:MM SAMPLE TUBE: XXXX POSITIVE</p>	<p>BIONOVA MiniBio RAPID READOUT INCUBATOR SERIAL NUMBER: XXXX XXX TICKET #: 000004 PROGRAM: X h / XX °C DATE: DD/MM/AA START TIME: HH:MM READOUT TIME: HH:MM SAMPLE TUBE: XXXX NEGATIVE</p>	<p>BIONOVA MiniBio RAPID READOUT INCUBATOR SERIAL NUMBER: XXXX XXX TICKET #: 000004 PROGRAM: X h / XX °C DATE: DD/MM/AA START TIME: HH:MM READOUT TIME: HH:MM SAMPLE TUBE: XXXX CANCELED</p>
POSITIVE	NEGATIVE	CANCELED

### Remaining incubation time

The Bionova® MiniBio allows verifying the remaining incubation time in every reading position by ticket printing. Whenever a reading process is in course, press and hold the button  for 3 seconds, a ticket will then be printed informing reading position, incubation program and remaining incubation time.

To verify remaining incubation time of 2 or more readings in course press 2 buttons  simultaneously; the device renders a ticket informing remaining incubation time of all positions.

<p>BIONOVA MiniBio RAPID READOUT INCUBATOR SERIAL NUMBER: XXXX XXX DATE: DD/MM/AA TIME: HH:MM</p> <p>TUBE 1: PROGRAM: X h / X° C REMAINING TIME: HH-MM h</p>
--

### Positive Control

The positive control should belong to the same batch of the processed SCBI.

## Instructions for use

Use a non-sterilized biological indicator as a positive control every time a processed indicator is incubated.

Incubate the processed biological indicator and the indicator used as positive control as long as specified in the instructions.

Press the cap to seal the tube and crush the ampoule, make sure the media completely wets the carrier. Write a C (for control) and the date on the label. The positive control can be placed in any position of the Bionova® MiniBio incubator. The positive control should belong to the same batch of the processed SCBI.

It is a good practice to use a positive control, this helps ensuring:

- Correct temperature is reached.
- Viability of spores has not been altered due to improper storage temperature, humidity, or proximity to chemicals.
- Capability of the media to promote rapid growth and fluorescence generation.
- Proper functioning of Bionova® MiniBio incubator.

### Interpretation of results

#### Rapid Readout

The indicator used as positive control should show a positive result (red light ⊗ ● ). The processed SCBI results are not valid until the biological indicator used as positive control shows a positive result ⊗ ● . A processed SCBI with a positive result, indicates a failure in the sterilization process. A negative result, (green light ⊕ ● ) after 20 min. (60 °C), 30 min. (60 °C), 1 h. (60 °C), 2 hs. (60 °C), 3 hs. (60 °C) 4 hs. (37 °C) according to the used program, indicates that the sterilization process was effective.

Take action on any positive result immediately. Always retest the sterilizer and do not use the sterilizer for processing loads until 3 consecutive SCBI results are negative.

#### Visual Readout

The Rapid readout system allows the confirmation of results by color change after 48 hours or 7 days incubation analysis (if 7-day readout is made, an humidified incubator will be required to avoid media dry out). If the sterilization process was not successful, the growth media contained in the indicator will turn to yellow, as it is shown in Instruction of Use of SCBIs. If

the sterilization process was successful, the media will remain unchanged.

The color of the growth media in the SCBI used as positive control must turn to yellow during the process of incubation for the results to be valid. Record the positive ones and discard them immediately as shown in the biological indicators instructions for use.

### Disposal

Dispose SCBIs after use, according to your healthcare and safety regulations. Positives SCBIs can be sterilized (see SCBIs instructions for use). It is not possible to reuse a SCBIs.

### Audible alarm

An audible alarm will sound every time a positive result is detected in the Bionova® MiniBio incubator. The alarm allows the user to immediately detect a positive result without the need to visually control the equipment. The alarm can be cancelled by pressing the button

### Cancellation of a reading

When a SCBI is removed from its position during incubation process, an audible alarm will sound to indicate that the tube should be returned to the corresponding position.

If the indicator is not re-introduced into the reading position after 10 seconds of being withdrawn, the reading will be cancelled and the cancelation will be informed through the printing of a ticket.

### Temperature monitoring

The Bionova® MiniBio incubator contains an internal temperature control. If the temperature falls outside of the specified range (37±2) °C or (60±2) °C, the blue temperature led light will start to twinkle.

Temperature can be externally monitored by using a Bionova® TB-IC1020 thermometer or other similar. The Auto-Reader MiniBio has a special hole for placing the thermometer which is located at the right side of the device.

# Instructions for use

## Thermal paper specifications

Recommended paper: JUJO AF50KSE3 or similar (order code ICTP).

Paper width : 58 mm

Maximum paper thickness: 60 g/m2

Maximum diameter size: 23 mm

## Replacement of the paper roll

! The printer uses thermal paper rolls.

1. To replace the paper roll, pull the handle of the printer's door. Open the printer's cover and remove the spent roll.

2. Place the new paper roll with the outer side up.

3. Close the printer cover by pressing on the sides of the lid.

**NOTE:** You can check the correct paper replacement by pressing  , this button will force the advance of the paper a few centimeters.

## Reprinting of results

The incubator allows reprinting of the last 3 results by fluorescence readout. For printing the results, press the button  for 3 seconds.

## Setup Mode

Bionova® MiniBio has a setup mode from which, the equipment time zone and the print language can be set up. To enter Setup mode, turn on the incubator while holding down the button . The red and green LEDs in the three positions will remain on, indicating that the device is in this mode. The printer will render a ticket indicating that it is in Setup Mode.

To change the time zone and the print language, follow the instructions below. When finished, restart the computer to exit Setup mode.

**NOTE:** Remember that it is possible to synchronize the time with the Bioupdate® software and Bionova® Traceability Software.

### Modification of time zone

The time zone modification function allows changing the time zone of the device to suit the time zone of your country.

To make this change, the incubator must be started in Setup mode (see **Setup Mode**), then press the button .

The current time will be printed to set a reference.

To subtract hours, press the button  as many times as necessary. To add hours, press the button  as many times as necessary. After 2 seconds, a ticket will be printed informing the selected time.

To confirm the change hold down the button  for 3 seconds. To discard the changes, unplug the incubator from the power source.

### Modification of printing language

This functionality allows changing the language in which the incubator printer renders the tickets.

To make this change, the device must be started in Setup mode (see **Setup Mode**), then press the button .

## Additional features

The current language and list of available languages will be printed to establish a reference. Press the button  to scroll up and to  scroll down. After 2 seconds, a ticket will be printed informing the selected language.

To confirm the change, press and hold the button  for 3 seconds. To discard the change, unplug the incubator from the power source.

### Time setting mode

#### Modification of the time using the Bioupdate® Software

Connect the auto-reader via the USB port to a PC and start-up the program. Select the Bionova® MiniBio Auto-reader and press the Clock Sync button, the auto-reader will be synchronized with the PC's clock. Synchronization can only be done as long as there is not an ongoing reading.

#### Change the time using the Bionova® Traceability Software

Connect the auto-reader via the USB port to a PC and start-up the program. The Bionova® MiniBio Auto-reader will be synchronized with the PC's clock. Synchronization can only be done as long as there is not an ongoing reading.

### Cleaning and maintenance

#### Cleaning and decontamination of external surfaces

Disconnect the power supply cable and the USB cable from the incubator. If the device is hot, wait until it has cooled down before handling it.

Clean the external surfaces of the device using a microfiber cloth moistened with a solution of mild dish washing detergent and water. Wring the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the incubator. Afterwards, moisten a clean microfiber cloth with water only, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the Incubator to air dry for at least 1 hour before connecting the power supply cable or the USB cable again.

This cleaning procedure can be followed whenever considered appropriate.

This cleaning procedure must be followed every time a spill occurs on any of the external surfaces of the incubator.

If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.

Do not clean the internal parts of the device.  
Do not pour or immerse the device into any liquid.  
Do not allow any liquid to run inside the device during its cleaning.

#### Maintenance

Bionova® MiniBio does not require routine maintenance.

### Additional features

#### Firmware update

Bionova® MiniBio Auto-reader Incubator allows the update of its firmware (program inside the Incubator that controls and defines its different features) by using the firmware update utility Bionova® Bioupdate. Bionova® Bioupdate connects to the internet to verify, download, and install, the latest firmware version available for your incubator. The update process only lasts a few seconds and is carried out without the loss of any Incubator's data. Please visit [www.terragene.com.ar/software](http://www.terragene.com.ar/software) to download Bionova® Bioupdate.

#### Update process

Before updating the device, make sure that the Bionova® Traceability Software is closed. Power Off the Incubator for five seconds, afterwards Power On the Incubator and follow the next steps:

1. Using the included USB cable, connect your Bionova® MiniBio to a PC where Bionova® Bioupdate has been previously installed. Make sure the PC has a working internet connection.
2. Run Bionova® Bioupdate.
3. Select the Incubator from the list and press the Start button to update.
4. Wait until the software indicates that update process has been completed. The Incubator will print an update confirmation ticket. If it was already up to date, the ticket will not be printed.

## Additional features

**NOTE:** This process must be repeated every time that a new firmware version is available, this will be informed in the Terragene® S.A. website.

### Bionova® Traceability Software

#### Managing results with Bionova® Traceability Software

Bionova® Traceability Software is a proprietary software designed by Terragene® to manage and store the results of Self-contained biological indicators and Hygiene Monitoring System Indicators in a secure and user-friendly way.

Bionova® Traceability Software allows matching the information of an Indicator result with the sterilizer or washing machine used, the operator, cycle characteristics, and all relevant information for allowing each institution achieve reliable documentation compliance and safe-keeping. All saved information is stored in a password protected encrypted database to assure its integrity. Furthermore, all previously saved information can be reprinted at any time to generate a hard copy of the results.

Visit [www.terragene.com.ar/software](http://www.terragene.com.ar/software) to download the latest version of Bionova® Traceability Software, its User Manual, and to check its newest features and capabilities.

#### Using Bionova® Traceability Software

To use Bionova® Traceability Software along with your incubator, please download the software from our webpage and install it in a compatible Personal Computer (PC). The PC minimum requirements are listed in the Traceability Software User Manual. After the installation is complete, connect your Bionova® MiniBio via the USB port to the PC and run Bionova® Traceability Software.

## Troubleshooting

Fault	Possible cause	Action
The incubator does not start.	Power supply is not connected.	Check that the power supply is connected to a proper mains. Check that the DC plug of the power supply is connected to the rear of the Incubator.
The Incubator informs an error in an incubation position during "Autotest".	A indicator is placed in the incubation position during the Autotest.	Check that every incubation position is empty during the Autotest.
The Incubator informs an error in an incubation position during "Autotest".	Dust particles might be obstructing the readout mechanism.	Avoid using the Incubator in dusty environments. Use air to dust-off inside a position. Do not insert solid objects. Once cleared, restart the Incubator.
An incubation can not be run in a SCBI incubation position (position's red light is on).	Position disabled. Error in that position during "Autotest".	Make sure that the position is empty when starting the Incubator. Once cleared, restart the Incubator.
The Incubator does not run an incubation in any position.	Incubation temperature not stable.	Wait until temperature is stable before any incubations.
Is not possible to change temperature and/or time setting.	Ongoing reading.	Wait for any incubation to complete, and try again.
The printer does not print (blue attention light blinks quickly).	The printer cover is not tightly closed. Printer without paper.	Check that the cover is tightly closed. Place a new paper roll in the right direction.
The printer releases unprinted paper.	Paper roll is not placed correctly.	Place the paper according to the directions on the <b>Replacement of the paper roll</b> section.
The Incubator can not be updated.	Bionova® Traceability Software is being run.	Close Bionova® Traceability Software and restart the Incubator.
the Incubator is not detected by Bionova® Traceability Software.	the Incubator is turned off / incubator is not connected to the PC.	Follow the directions on the <b>Start-up</b> section to set-up the device properly.

## Warranty

Terragene® S.A. guarantees both the quality of the product material components and the quality of its manufacturing process. Should any material or manufacturing faults be detected within the warranty period (1 year from the time of purchase), the only obligation of Terragene® S.A. will be product repairing or substitution.

### Limitation of liability

Terragene® S.A. shall not be held liable for any loss or damage that may result from the unsuitable use of the equipment, negligence or user's full responsibility.

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