TERRAGENE

Product Name

BT90 – F1 - Rapid Readout Colorimetric System. Self-contained Biological Indicator for plasma or vapor Hydrogen Peroxide sterilization processes.

Usage

Monitoring plasma or vapor Hydrogen Peroxide (H₂O₂) sterilization processes.

Applicable Regulation

EN ISO 11138-1:2007 and IRAM 37102-1:1999.

Product Classification

Class 1, according to risk.

Authorization

Designed under Quality Systems ISO 9001:2008 and ISO 13485:2012. ANMAT (Argentinean National Administration of Drugs, Food and Medical Technology) PM-1614-1.

Product Characteristics

Polypropylene random copolymer tube: 51.0 mm high x 8.5 mm external diameter. Wall thickness: 0.5 mm.

Polypropylene separator: 7.0 mm diameter.

Polypropylene cap: 10.0 mm external diameter, 16.5 mm high. Wall thickness: 0.5 mm

Glass ampoule: 23.0 mm high. External diameter: 6.5 mm. Wall thickness: 0.2 – 0.3 mm.

Culture medium: 0.2 ml, **light purple** color.

Cap filter: polyethylene, 16.0 mm diameter.

10⁵ or 10⁶ *Geobacillus stearothermophilus* ATCC 7953 spores per vial.

Positive growth results (failure in the sterilization process): culture medium turns to **green-yellow**, incubation at 60 °C.

Colorimetric final reading at 8 hours of incubation (sensitivity: 97 %). Measures every 30 minutes within the 8 hour-incubation are recommended.

D-Value: 2 mg/l H₂O₂, 45 °C.

Environmental conditions during manufacture

Room temperature 15–30 °C, RH 30–80 %. Sterility conditions are necessary only during inoculation process performed in laminar flow.

Storage conditions

T = 15–30 °C, RH 35–60%, keep in a dark place.

Transportation Conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

24 months.

Packing

100 units per box.

Packing information: product code and description, process for intended use, presentation, classification according to regulation, manufacturer information and data on box's label. Note: manufacture date is calculated by subtracting 24 months to the expiration date.

Labelling

On product: 17.0 mm x 33.0 mm polypropylene label. Printed in black. 1.5 mm chemical indicator line, printed with Hydrogen Peroxide reactive ink (Color change to green). Product code and batch number, process for intended use and bacterial strain.

On product's box: product code, batch number, bacterial load, manufacture and expiration date.

Possible target markets

Healthcare and Industry.

Other important information (optional)

It is advisable to incubate at 60 °C in BIONOVA® IC10/20 incubator.
Read product's directions for use thoroughly before use.

Note: When necessary, limits of each product's parameters should be specified.

Description of identified and/or predictable risks and situations which could represent a risk for the product.

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry heat, Radiation or any sterilization process other than Hydrogen Peroxide.

The positive biological indicator can be autoclaved at 121 °C for at least 20 minutes, at 132 °C for at least 15 minutes in gravity displacement steam sterilization, or at 134 °C for at least 10 minutes in vacuum steam sterilizer.

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