# BDA4

## Indicador Químico

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### Uso previsto

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*Bowie & Dick Test Sheet*

### Usage

Bowie & Dick Test Sheet, for evaluation of air displacement and steam penetration in vacuum assisted steam sterilizers. Bowie & Dick cycles of 3.5 minutes at 134 °C or 4 minutes at 132°C.

### Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.  
ISO 11140-1:2014 and ISO 11140-3:2007

### Characteristics

Type 2 Chemical Indicator.  
A4 size sheets, 80 g/m<sup>2</sup>.  
Initial color: **yellow**  
Final color: **dark brown/ black**

#### NOTE:

Air removal or steam quality and penetration problems are evidenced as a heterogeneous color change of the chemical indicator, with lighter regions in the center of the sheet.

100 % Lead Free

### Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

### Storage conditions

T = 10-30 °C, RH 30-80 %, keep out of direct light.

### Transportation conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

**Shelf-life**

5 years

**Packing**

50 sheets per box.

Packing information: product code and description, presentation, process for intended use, classification according to regulation, manufacturer information and data on label.

**Labelling**

Only on product's box: product code, color change, batch number, expiration date, barcode and datamatrix code.

**Possible target markets**

Healthcare, Food, Pharmaceutical and Medical Industries.

**Other important information**

Read product's instructions for use thoroughly before use.

**Description of identified and/or predictable risks which could represent a risk for the product**

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat, Radiation or any sterilization processes other than Steam.

Do not use this product for air removal monitoring of gravity displacement Steam sterilizers.

Do not use the sterilizer until the color change of the test sheet is according to the expected result, as detailed in the product reference.