# IKW4

## Tinta indicadora

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# IKW4

## Indicator ink

*For washing efficiency tests in washer-disinfectors*

## Usage

Monitoring the washing efficiency in thermal disinfection washing machines and in ultrasonic washing machines operating at frequencies equal to or greater than 35 kHz, between 20 ° C and 65 ° C, with detergents suitable for this process.

## Authorization

Designed under Quality Management System standards ISO 9001:2008 and ISO 13485:20012.

## Characteristics

- Washing Indicator Ink contained in a glass bottle with applicator brush. Volume: 10 ml
- Initial Color: fuchsia

Result: elimination of fuchsia organic substance.

**Important:** IKW4 Indicator Ink should be used as a validation tool for washing processes. Its application, the amount placed and sites where adhered depend on the user. It is recommended for use in difficult-to-wash areas of metallic surgical materials.

## Environmental conditions during manufacture

Room temperature 15-30 °C, RH 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

## Storage conditions

T = 10-30 °C, RH 30-80 %, keep in a dark place.

## Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

## Shelf-life

12 months.

### **Packing**

2 units per box.

Packing information: Code, product description, process in which is used, units per box, batch number, expiration date, instructions for use and manufacturer information.

On product's packaging: volume, product code, process in which it is used, batch number, expiration date.

Weight per box: 96,6 g

### **Labelling**

On the product's box: batch number, barcode and expiration date.

### **Possible target markets**

Healthcare and Industry.

### **Other important information (optional)**

Read product's directions for use thoroughly before use.

Shake before use. Keep closed. Industrial or professional use only.

### **Description of identified and/or predictable risks which could represent a risk for the product**

Do not open until use.

Not for domestic or cosmetic purposes.

Do not wet.

Do not use after expiry date.