# BT20

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# BT20

## Self-Contained Biological Indicator.

*For Steam sterilization processes.*

### Usage

Monitoring vacuum-assisted and gravity air-displacement Steam Sterilization processes at 121-135 °C.

### Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

ISO 11138-1:2017 and ISO 11138-3:2017; IRAM 37102-1:1999 and IRAM 37102-3:1999.

### Classification

Class 1, according to risk.

### Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

### Characteristics

Polypropylene tube: 45.2 mm high, 8.5 mm external diameter. Wall thickness: 0.5 mm.

Polypropylene cap: 10.6 mm diameter, 13.0 mm high. Wall thickness: 0.9 mm.

Cap filter: medical grade paper, 60 g/m<sup>2</sup>, 17.0 mm diameter.

Glass ampoule: 35.0 - 40.0 mm high. External diameter: 6.8 mm. Wall thickness: 0.2 - 0.3 mm.

Culture medium: 0.5 - 0.7 ml, **purple** color.

Spore carrier: filter paper: 7.0 mm diameter, 160 g/m<sup>2</sup>.

10<sup>5</sup> or 10<sup>6</sup> *Geobacillus stearothermophilus* ATCC 7953 spores per vial.

Positive growth results (failure in the sterilization process): culture medium turns to **yellow**, incubation at 60 °C for 24 hours (maximum).

D-Value: Not lower than 1.5 minutes at 121 °C. Another D-value is informed at 132 °C and 135°C.

Z-Value is informed.

### Environmental conditions during manufacture

Room temperature 15-30 °C, RH 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

### Storage conditions

T = 10-30 °C, RH 30-80 %, keep in a dark place in its original box.

### Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

### Shelf-life

2 years.

### **Packing**

30 units or 100 units per box.

Packing information: product code and description, process for intended use, presentation, regulation, bacterial strain, storage conditions, manufacturer information and data on box's label.

### **Labelling**

On product: 17.0 mm x 33.0 mm polypropylene label printed in black. 1.5 mm chemical indicator line, printed with Steam reactive ink (color change to brown). Product code, batch number, expiration date, process for intended use and bacterial strain.

On product's box: product code, batch number, bacterial load, manufacture and expiration date, barcode and datamatrix code.

NOTE: manufacture date is calculated by subtracting 24 months to the expiration date.

### **Possible target markets**

Healthcare and Industry.

### **Other important information**

It is advisable to incubate at 60 °C in Bionova® IC10/20 incubator.

Read product's directions for use thoroughly before use.

### **Description of identified and/or predictable risks which could represent a risk for the product**

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat, Radiation or any sterilization process other than Steam.

The positive biological indicator can be autoclaved at 121 °C for at least 20 minutes, at 132 °C for at least 15 minutes in gravity displacement steam sterilization, or at 134 °C for at least 10 minutes in vacuum steam sterilizer.