# KPRO2-E250

## Sistema de Control de Higiene

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# KPRO2-E250

## Hygiene Monitoring System.

*Kit system for detecting residual protein on endoscopes and other cannulated instruments.*

### Usage

Designed to detect proteins on endoscopes and other reusable instruments with hard to reach internal channels or cannulated instruments.

### Applicable Regulation

ISO 15883-1 / HTM2030 / HTM 01-05

### Product Classification

Class 1, according to risk.

### Authorization


Designed under Quality Management System standard ISO 13485:2003/NS-EN ISO 13485:2012.

### Characteristics

- The system has a flexible high absorption swab SW250: 2.5 m long, 2 mm diameter absorption area + a reactive colorimetric solution. The SW250 swab can be introduced through the biopsy channel of the endoscope thus allowing sampling contaminants left after the cleaning process.
- Volume of solution: 0.440 ml.

- An instant and visual reading of a color change indicates the presence of detectable levels of protein. By comparing the color of the test against the table an estimation of the surface cleanliness can be made.

- KPRO2-E250 can detect as little as 1 µg of protein.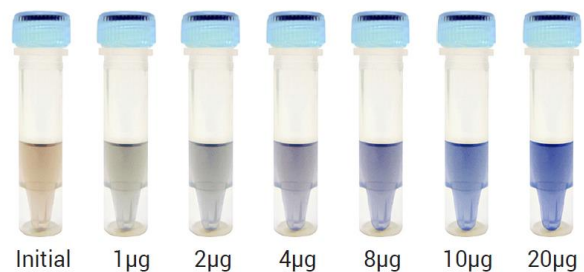

### Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

### Storage conditions

T = 2-25 °C, RH 30-80 %, store in dark place.

### Transportation conditions

Storage conditions should be strictly followed.

Transport in closed and strengthened boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

### Shelf-life

24 months when stored refrigerated (2°C – 8°C) and 6 months when stored at room temperature up to 25 °C.

**Packing**

50 units per box + 50 SW250 swabs + 4 moisturizer tubes for swabs.  
Packing information: product code and description, process for intended use, classification according to regulation, manufacturer information and data on label.

**Labelling**

On product's box: Product code and description, batch number and expiration date.

**Possible target markets**

Healthcare, Food, Pharmaceutical and Medical industries.

**Other important information (optional)**

Read product's directions for use thoroughly before use.

Note: When necessary, limits for each product's parameters should be specified.

**Description of identified and/or predictable risks which could represent a risk for the product**

Do not reuse.

Do not reuse.

Do not touch the swab with your hands. Wear gloves, surgical mask and other hygiene measures.

Do not freeze.

Although PRO2 has high sensitivity, it is not a microbiological monitoring device, and surfaces may not be considered to be sterile based on the results obtained.