Bionova® MiniPro Auto-reader

Operating relative humidity
Humedad relativa de funcionamiento
Umidade relativa de funcionamento
Umidità relativa d'esercizio

Operating temperature
Temperatura ambiente de funcionamiento
Temperatura ambiente de funcionamento
Temperatura di esercizio

Caution, Warning, Attention - Refer to the instructions for use
Atención - Consulte las instrucciones de uso
Atenção - Consulte as instruções de uso
Attenzione, Avvertimento, Prendenza - Fare riferimento alle istruzioni per l'uso

Caution! Hot surface
Atención! Superficie caliente
Atenção! Superfície quente
Attenzione! Superficie calda

Important
Importante
Importante
Importante

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EN Bionova® MiniPro Auto-reader

Product description

Bionova® MiniPro Auto-reader has been designed for the automatic readout of Terragene® Chemdye® Hygiene Monitoring System Indicators (Protein Pens) appropriate for surface cleaning and contamination control applications.

Bionova® MiniPro Auto-reader incubates at 60 °C and performs a quantitative analysis over the Chemdye® PRO1 MICRO Protein Pen, adjusted by a BSA (Bovine Serum Albumin) reference curve. Bionova® MiniPro allows the incubation and readout of 3 independent Protein Pens simultaneously. Bionova® MiniPro quantitative analysis has a detection limit of 1 µg and a sensitivity of 0.3 µg. The results are indicated with a resolution of 0.1 µg, and their accuracy is ± 0.5 µg (IC95 %), in the range of 0 to 5 µg and ± 0.7 µg (IC95 %), in the range of 5 to 10 µg with a range of 10 µg. Bionova® MiniPro Auto-reader also provides a printed ticket each time an automatic readout is completed for recording the results. This allows for easy result management, for documentation compliance and safekeeping. **Important:** Do not place a new Protein Pen until the Temperature is stable and the Temperature Stability Indicator stops blinking.

Indications for Use
Terragene® Bionova® MiniPro incubator incubates at 60 °C and reads the Terragene® Chemdye® Hygiene Monitoring System Indicators at the times prescribed in the User Manual.

Safety Information

To avoid risks and/or damaging the device:

_For indoor use only.
_Do not place the Auto-reader in a room exposed to direct sunlight or to high luminous intensity lamps.
_Do not place the Auto-reader near devices that emit strong electromagnetic fields.
_Do not use the Auto-reader on leaned surfaces or on surfaces that are subjected to shocks, vibrations, temperature or high relative humidity.
_Disconnect the power cord before cleaning.
_Do not use abrasive or corrosive cleaners or disinfectants.
_Do not immerse into any liquid. Do not pour any liquid inside.
_Make sure the Auto-reader is connected to an appropriate electrical mains outlet socket.
_Use only the included power supply (AC power adapter), power supply's AC plug, power supply cords, and USB cable. Verify that all the included elements are in good condition on daily basis. If any of them is damaged, discontinue their use. The use of cables, adapters, cords, and/or power supplies different from the ones included may cause fire, electrical shocks, or even physical injuries.
_Do not attempt to repair the Auto-reader by yourself. That could lead to major and irreversible damages to the device. In case of device malfunction, contact your local distributor for further assistance.
_Do not plug any devices into the Incubator's USB port other than a personal computer (PC). The PC has to be compliant with IEC60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its USB ports. Ask a qualified technician to verify device compatibility. Attaching any other device to the USB port may damage the incubator and may not be safe for the user.

To reduce the risks associated with contaminated surfaces with protein levels outside the user-defined threshold:
_Please read, understand, and follow the Instructions for Use of each Protein Pen before its incubation.
_Do not remove the Protein Pen before the Auto-reader reports the final readout result. Check the printed ticket.

To avoid a potentially hazardous situation:
_Avoid contact with the hot metal block inside each incubation position.
_Do not insert your fingers, or any other element, inside the incubation positions.
_Place only compatible indicators inside the incubation positions.

Important:
_Do not use this product in a manner not specified by Terragene® S.A., otherwise the protection provided by the product might be affected.
_Only Terragene® S.A. authorized personnel can access or service the internal components of the Auto-reader. Parts or components inside the Auto-reader should not be manipulated by the user.

Operating conditions

Input parameters	Operating Conditions	Units	
Voltage range	(100-240)	AC Volts	
Frequency	50/60	Hertz	
*Corriente	0.5	Amperes	
Output parameters	Values	Units	
Voltage range	12	DC Volts	* 1 Amper in USA and Canada
**Current	2	Amperes	** 3 Amperes in USA and Canada

Terragene® S.A. recommends the use of UPS instead of voltage stabilizers, since they fulfill two functions: to stabilize and maintain the energy during a power outage.

Environmental operating conditions	Operating Conditions	Units	
Altitude	3500 (máx.)	Meters	
Operating temperature	10-30	Celsius	
Relative humidity	30-80	%	
Installation/overvoltage	Category II		
Degree of contamination	2		
Storage temperature	10-30	Celsius	
Voltage	12	DC Volts	

Regulatory compliance

Bionova® MiniPro Auto-reader complies with the following standards and directives:

Electrical Safety	IEC 61010-1 <p>IEC 61010-2-010 Low Voltage Directive 2014/35/EU</p>	
Electromagnetic Compatibility (EMC)	EN 61326-1 <p>EMC Directive 2014/30/EU</p>	
European Commission	RoHS Directive 2011/65/EU <p>WEEE Directive 2012/19/EU</p>	

Bionova® MiniPro Auto-reader do not represent photobiological risk and do not generate dangerous optical radiation in any of its normal operation conditions as per the requirements of IEC 62471 Standard. Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

Cleaning and maintenance

Cleaning and decontamination of external surfaces

Disconnect the power supply cable and the USB cable from the Auto-reader. If the device is hot, wait until it has cooled down before handling it.
Clean the external surfaces of the device using a microfibr cloth moistened with a solution of mild dish washing detergent and water. Wring the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the Auto-reader. Afterward, moisten a clean microfibr cloth with water only, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the Auto-reader to air dry for at least 1 hour before connecting the power supply cable or the USB cable again. This cleaning procedure can be followed whenever considered appropriate. This cleaning procedure must be followed every time a spill occurs on any of the external surfaces of the Auto-reader.
If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.

! Do not clean the internal parts of the device.
Do not pour or immerse the device into any liquid. Do not allow any liquid to run inside the device during its cleaning.

Maintenance

MiniPro Auto-reader does not require routine maintenance.

Warranty

Terragene® S.A. guarantees both the quality of the product material components and the quality of its manufacturing process. Should any material or manufacturing faults be detected within the warranty period, the only obligation of Terragene® S.A. will be product repairing or substitution.

Term

The warranty period for electronic products marketed by Terragene® will be 1 (one) year from the date of first use of the product, and may never be extended beyond the product's useful life -5 (five) years from its manufacture-.

Limitation of liability

Terragene® S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

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EN Instructions for use

Setting Up your Auto-reader
Place the Bionova® MiniPro Auto-reader on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Auto-reader in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Auto-reader to the closest wall. Do not move the Auto-reader periodically or during its use. Connect the Auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

Using your Auto-reader
1| Turn on your MiniPro Auto-Reader by connecting the AC plug of the power supply to the mains and the jack at the other end of the power supply to the rear of the Auto-Reader. (See image 1)
2| Wait until the temperature stability indicator stops blinking. Press the Pro button of the selected position for 1 second to enter the reading mode. (See image 2)
3| Wait until the qualitative indicator of protein concentration corresponding to 1 µg stops blinking. Place the protein pen in the corresponding position. Press the Pro button again for 2 seconds to start the incubation. (See image 3)
4| Wait until the incubation is completed. The position status indicator light will turn green to indicate a negative result or red to indicate a positive result. For the PRO1 MICRO indicator, the concentration of the protein detected will also be indicated. If a positive result is detected, press the alarm button to cancel the alarm. Do not remove the indicator until the ticket with the result of the incubation is printed.
Note: To modify the protein concentration threshold to determine a positive result for the PRO1 MICRO Indicator, please see the user's manual.
Disposal: Dispose of the indicators in accordance with your country's waste-disposal guidelines.

Connectivity
To connect your MiniPro Auto-reader to a PC via a USB port, connect an end of the USB cable (included) to the rear part of the incubator and the other end to the USB port of your PC. Connecting the Auto-reader to a PC allows the use of the Bionova® Traceability. Please see the User's Manual for further information.

Product Compatibility and Incubation programs

Bionova® MiniPro Auto-reader Incubation Programs
The following incubation programs are available for your Bionova® MiniPro Auto-reader.

Temperature	Time	Incubation Program
60 °C	PRO (10 min.)	PRO at 60 °C

⚠ Follow each Indicator's Instructions for use before its incubation.

Bionova® MiniPro Auto-reader compatible indicators
We develop new indicators for extending the features of our Auto-readers regularly. Please visit www.terragene.com/MINIPRO to access the Compatibility Matrix for Terragene® Auto-readers. The Compatibility Matrix specifies all compatible indicators developed for your Auto-reader up to date. Stay tuned for new and exciting updates!

Troubleshooting
Refer to the user manual for a detailed troubleshooting routine. Please visit www.terragene.com/MINIPRO or scan the QR code to access the Auto-reader user manual.

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