Auto-reader  
for Rapid, Super Rapid, Ultra Rapid  
Biological Indicators  
and Hygiene Monitoring  
systems

*Innovation in technologies for Infection Control*


IC10/20FR

TERRAGENE®

Rev.26 | November 2018
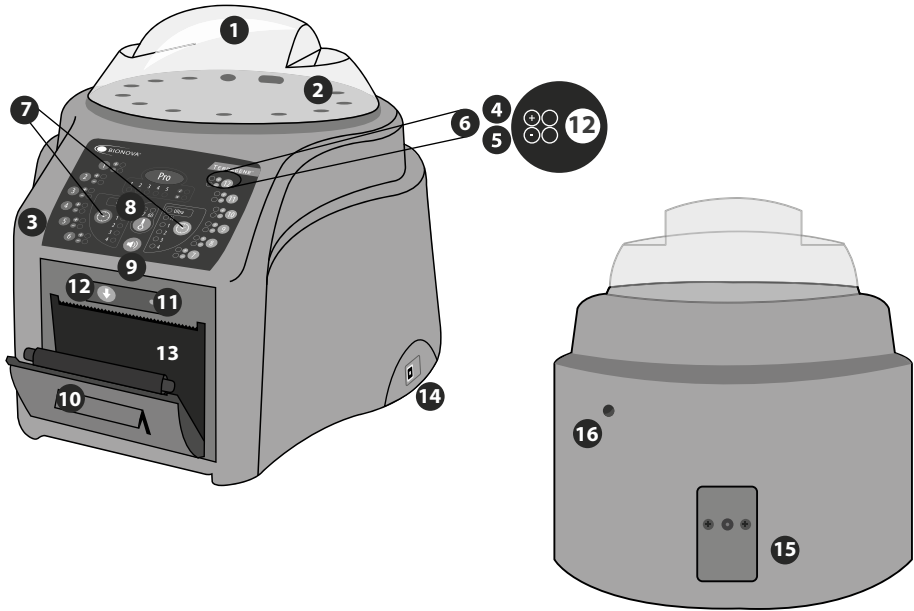
## Constituent parts

### Description of use

Bionova® IC10/20FR Auto-reader has been designed for the incubation of the following Self-Contained Biological Indicators (IBs):

- Rapid Readout Self-Contained Biological Indicators: BT110 for Ethylene Oxide sterilization control, BT220 and BT221 for Steam sterilization control, BT102 for Formaldehyde sterilization control and BT95 for Plasma or Vaporized Hydrogen Peroxide sterilization control.
- Super Rapid Readout Self-Contained Biological Indicators: BT222 and BT223 for Steam sterilization control and BT96 for the control of Plasma or Vaporized Hydrogen Peroxide sterilization control.
- Ultra Rapid Readout Self-Contained Biological Indicators: BT224 for Steam sterilization control.

Microorganisms contained within the Biological Indicators vary depending on the sterilization process for which they have been designed, therefore, incubation temperatures differ depending on the indicator. Indicators for Ethylene Oxide (EO) sterilization control, should be incubated at 37°C, while those involved in the monitoring of the rest of the sterilization processes should be incubated at 60°C.

Simultaneous incubation of IBs for EO processes control within IBs monitoring the other processes, is not possible.

Bionova® IC10/20FR Auto-reader allows easy and rapid

detection of positive and negative Biological Indicators. Six incubation programs are available: 4 hours at 37°C, 3 hours at 60°C, 2 hours at 60°C, 1 hour at 60°C, 30 minutes at 60°C and *Instant/Ultra* (20 min.) at 60°C. A positive result can also be evidenced by culture medium color change to yellow, read the instructions for use of each indicator for more information. The choice to perform or not the extended incubation to confirm by color change depends on the internal protocols of each laboratory or hospital. The auto-reader allows selection of incubation time programs for two zones in an independent way. Zone 1 includes positions 1 to 6, and zone 2 comprises positions 7 to 12.

The Bionova® IC10/20FR Auto-reader contains a special cavity to run quantitative readings of the ChemDye® PRO1 System, destined to surface cleaning and contamination control.

Bionova® IC10/20FR auto-reader not only allows the detection of results, but also provides a printed ticket to record them. In addition, the auto-reader can be connected to computers by USB. This allows linking the results with the *Bionova® Traceability Software* for Automatic Reading and Traceability of Rapid, Super Rapid and Ultra Rapid Biological Indicators and Hygiene Monitoring System. To download the latest version of the software, go to: <http://www.terragine.com.ar/productos/infection-control/incubators/software-downloads/?lang=en>, from there you can also download the user manual.

## References

- 1 Protective cover.
- 2 Incubation area for 12 BIs. Protein pen detection. BIs Ampoule crusher.
- 3 Control Panel.
- 4 Red light/Positive indicator/Position Light.
- 5 Green Light/Negative indicator/Position Light.
- 6 Reading position number.
- 7 Program selector.
- 8 Temperature Selector.
- 9 Alarm Cancellation Button.
- 10 Thermal printer.
- 11 Printer's light.
- 12 Paper's traction button.
- 13 Cavity for paper.
- 14 USB port.
- 15 Input for power source plug (12V DC).
- 16 Hole for external temperature control.

## Symbols

-  80 %  
30 % Operating relative humidity.
-  30 °C  
10 °C Operating environmental temperature.
-  Caution, warning.
-  Caution: Risk of electric shock.
-  Caution: Hot surface.
-  Important, attention.
-  Manufacturer.
-  Serial number.
-  Direct current.
-  Product denomination.

### To avoid risks and damaging the equipment

- Do not place the auto-reader in an environment exposed to direct sunlight or high intensity light lamps.
- Do not place the auto-reader near devices that emit electromagnetic fields.
- Do not use the equipment on vibrating surfaces.
- Do not pour any liquid inside.
- Do not immerse into any liquid.
- Use indoors only.
- Disconnect the power cord before cleaning.
- Do not use abrasive, corrosive cleaners or disinfectants.
- In case of technical fault, contact the manufacturer for support. Please, do not try to open or repair the auto-reader on your own, since this will imply the loss of product warranty and could lead to a major and irreversible damage.
- Make sure that the auto-reader is connected to a properly rate power cord.

### To reduce the risk of using incompletely sterilized loads:

- Make sure to use each BI with its incubation and reading program:
- BT110 (green top) with 4 hs. (37 °C) program.
- BT220 (brown top) with 3hs. (60 °C) program.
- BT221 (blue top) with 1 h. (60 °C) program.
- BT222 (violet top) with 1 h. (60 °C) program.
- BT223 (light blue top) with 30 min. (60 °C) program.
- BT224 (aquamarine top) with 20 min at 60 °C *Instant/Ultra* program.
- BT102 (red top) with 2 hs. (60 °C) program.
- BT95 (white top) with 2 hs. (60 °C) program.
- BT96 (grey top) with 30 min at 60°C program.

- Check that culture medium completely wets the spore carrier.
- Do not remove the BI until the equipment reports the final readout result, check the impression of the ticket.

### To avoid the risk of injury, because of glass fragments generated after the glass ampoule inside the BI tube is crushed:

- Cool the BI during the indicated time before crushing the ampoule.
- Do not handle the BI excessively since this might cause the glass ampoule to burst.
- Wear safety glasses and gloves when removing the BI from the sterilizer.
- Wear safety protective glasses and gloves to crush the ampoule and to press the cap of BI.
- Do not use your finger for crushing BI.

### To avoid a potentially hazardous situation:

- avoid contact with the hot surface of the metal auto-reader block.

## Start-up

1-Place the auto-reader on a firm surface, free from vibrations, away from sunlight, currents of hot or cold air, chemicals and corrosive or flammable substances. Do not place the equipment in a way that disconnection of the plug from the power supply could be difficult. Leave a gap of at least 10 cm from the wall. Do not move the auto-reader periodically or during its use. Connect the auto-reader to a secure and stable electrical power source.

 Do not wet or heat the auto-reader. If liquid is spilled on the auto-reader, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

2-Turn on the machine by connecting the end of the AC power supply to the mains and then connect the other end of the source (plug) on the rear of the IC10/20FR unit. The printer's paper indicator light will turn on, if this light becomes intermittent, this means that the thermal paper must be loaded, or that the printer cover is not tightly closed. Otherwise, the light will remain stationary. The machine will start running the last time and temperature set-up.

 If the auto-reader has an out-of-date Firmware version, the settings might not be saved.

3-Synchronize IC10/20FR with local time using Bionova® software available at <http://www.terragene.com.ar/productos/infection-control/incubators/software-downloads/?lang=en>. For this, connect a PC to the USB plug of the auto-reader and start the program. The Bionova® IC10/20FR will be synchronized with the time and time zone in the PC. Synchronization can be done as long as there is not an ongoing reading. See other configuration methods in the *Time setting mode* section.

 All equipment is manufactured with the following settings:  
- Time zone: UTC +0:00  
- Printing language: English

For changing time zone, refer to *Time zone change mode* section. For changing printing language, refer to *Printing language* section.

4- Check that the incubation temperature is correct. To change it, press the button  for 3 seconds. Lights corresponding to 37 °C and 60 °C will start blinking, indicating that a temperature program must be selected. For temperature selection, press the button  to select 37 °C or 60 °C. After 4 seconds the temperature modification will be accepted.

For 37 °C, the IC10/20FR auto-reader will automatically select the 4 hrs. final reading program in both halves of the incubation area (1-6 and 7-12).

For 60 °C, *Instant/Ultra* (20 min.), 30 min., 1, 2 or 3 hrs. program can be selected. It's possible to select different programs for each half or the same for both, depending on the indicator's final reading characteristics. Check that the time setting is appropriate for the BIs

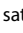
to be incubated in each zone.

To change the time program, press the button  of the correspondent zone as many times as needed to choose the right setting.

5- Wait for temperature of the machine to be stable. Once is stable, the indicator of the desired temperature will remain steady.

 Do not put any biological indicator in the reading positions until the auto-reader does not confirm temperature stability.

The auto-reader will keep the selected incubation program during the reading process. The system does not allow to modify the incubation program once the reading process has begun, unless the reading is finished.

6-Start readings once the blue temperature selection's light remains stable, this indicates that incubation temperature was reached. Once the selected temperature has been reached and remains stable, after turning on the equipment, a one-time automatic check of the Biological Indicators and Protein Pen incubation positions will be carried out. Indicating lights of each well will turn on bringing the result, a green light  will show a satisfactory result while a red light  will indicate error.

When the equipment detects an error in any position, a ticket indicating the day, time and position involved will be printed. To guarantee the reliability of the results, these positions will be deactivated, thus preventing their use. To evidence this, the red light  of that position will remain blinking indefinitely, in order to indicate that the position has been disabled.

 For a correct automatic check, the protective cover of the auto-reader should be in its place.

 Read BIs instructions for use. Before placing the vials in the auto-reader, press the top to seal the tube. Crush the ampoule (using the ampoule crusher within the box of BIs or the one located within the incubation area of the auto-reader) and make sure that the liquid has completely wet the spore carrier placed at the bottom of the biological indicator plastic tube.

 Do not remove or change placement of BI once the reading process has begun. If this occurs, results may be invalidated.

 Use a non-sterilized biological indicator as a positive control every time a processed indicator is incubated. For more information refer to the *Positive Control* section of this user guide.

7-Place the rapid BI in a reading position and wait for the result. Reading will start automatically. Immediately after placing the BI in a position, an alarm will sound indicating that reading has successfully began and the position LED will blink until the result is informed.

**NOTE:** If the auto-reader does not run automatically, it might indicate that the auto-reader temperature is not steady yet (check the temperature indicating light to

# Instructions for use

remain steady) or that the incubation position is disabled due to an error in the initial autotest.

8-If the incubator detects a positive result, the red light and the audible alarm will turn on. This will indicate that the sterilization process to which the BI was subject, has failed. 30 seconds after removing the IB from the incubation position, the alarm and the red light will turn off automatically and the position will be available to start a new reading.

The negative BIs will be informed with a green light in the corresponding position. This will indicate that the sterilization process to which it was subject has been successful. The green light will remain on for up to 30 seconds after the BI is withdrawn from the reading position.

The incubator will inform results within 20 min (60 °C), 30 min. (60 °C), 1 h. (60 °C), 2 hs. (60 °C), 3 hs. (60 °C) or 4 hs. (37 °C), according to the program selected. These periods of incubation are the time-limits used by the equipment to inform the fluorescence final readout.

9-Every time a positive result is detected, it will be informed through the printing of the corresponding ticket and with an audible alarm. The alarm can be cancelled by pressing the button .

10-To register the tickets, press paper's traction button, take the paper and press up for the paper to be cut with the serrated edge of the printer.

**NOTE:** If the paper runs out, printer's blue light will start blinking (see *Replacement of the paper roll in the printer*). The incubator will save the last 12 results, for re-printing, follow the *Reprinting of the last results* instructions.

BIONOVA IC10/20FR RAPID READOUT INCUBATOR SERIAL NUMBER XXXX XXXX PROGRAM XXh / XX°C DATE DD/MM/AA START TIME:HHMM READOUT TIME:HHMM SAMPLE TUBE: X NEGATIVE	BIONOVA IC10/20FR RAPID READOUT INCUBATOR SERIAL NUMBER XXXX XXXX PROGRAM XXh / XX°C DATE DD/MM/AA START TIME:HHMM READOUT TIME:HHMM SAMPLE TUBE: X POSITIVE	BIONOVA IC10/20FR RAPID READOUT INCUBATOR SERIAL NUMBER XXXX XXXX PROGRAM XXh / XX°C DATE DD/MM/AA START TIME:HHMM READOUT TIME:HHMM SAMPLE TUBE: X CANCELED
--	--	--

NEGATIVE

POSITIVE

CANCELED

## Positive control

The positive control should belong to the same batch of the processed BI.

Use a non-sterilized biological indicator as a positive control every time a processed indicator is incubated.

Incubate the processed biological indicator and the indicator used as positive control as long as specified in the instructions.



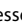
Press the cap to seal the tube and crush the ampoule, make sure the media completely wets the carrier. Write a C (for *control*) and the date on the label. The positive control can be placed in any well of the Bionova® IC10/20FR Auto-reader.

It is a good practice to use a positive control, this helps ensuring:

- Correct temperature is reached
- Viability of spores has not been altered due to improper storage temperature, humidity, or proximity to chemicals.
- Capability of the media to promote rapid growth.
- Proper functioning of Bionova® IC10/20FR Auto-reader.

## Interpretation of results
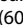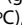
### Rapid Readout

The indicator used as positive control should show a positive result (red light  ). The processed BI results are not valid until the biological indicator used as positive control shows a positive result  . A processed BI with a positive result, indicates a failure in the sterilization process. A negative result, (green light  ) after 20 min. (60 °C), 30 min. (60 °C), 1 h. (60 °C), 2 hs. (60 °C), 3 hs. (60 °C) 4 hs. (37 °C) according to the used program, indicates that the sterilization process was effective.

Take action on any positive result immediately. A positive result indicates a sterilization process failure may have occurred. Determine the cause of the positive BI following current facility policies and procedures. Always retest the sterilizer and do not use the sterilizer for processing loads until 3 consecutive BI results are negative.

### Visual Readout

The Rapid readout system allows the confirmation of results by color change after 48 hours or 7 days incubation analysis (if 7-day readout is made, an humidified auto-reader will be required to avoid media dry out). If the sterilization process was not successful, the growth media contained in the indicator will turn to yellow. If the sterilization process was successful, the media will remain unchanged.

The color of the growth media in the BI used as positive control must turn to yellow during the process of incubation for the results to be valid. Record the positive ones and discard them immediately as shown in the biological indicators instructions for use.

## Disposal

Dispose BIs after use, according to your healthcare and safety regulations (see BIs instructions for use).

## Hygiene monitoring systems

### Chemdye® PRO1MICRO reading system

Chemdye® PRO1MICRO reading system consists of an incubation position that allows the quantitative analysis of the products destined to evaluate cleaning and contamination control.

# Instructions for use

IC10/20FR can run a quantitative analysis of PRO1MICRO Hygiene monitoring system, adjusted by a BSA (Bovine Seric Albumin) calibration curve. This analysis ensures the traceability of the control of surface cleaning.

The method has a detection limit of 1 µg and a sensitivity of 0.3 µg. The result is indicated with a resolution of 0.1µg, and its accuracy is ± 0.5µg (IC95%), in the range of 0 to 5µg, and ±0.7µg (IC95%), in the range of 5 to 10 µg with a range of 10 µg.

Readings will be able to initiate once the incubation temperature has been selected and is stable. The lights corresponding to the selected temperature (37 °C or 60 °C) will blink until the temperature is reached, and then stay stable. PRO1MICRO readings must be performed at 60 °C.

## Reading Process:

1-Press and hold the button **Pro** for 1 second to enter the reading program. A beep will sound. Then wait three seconds to confirm the selection. The auto-reader will emit two beeps indicating that a program has been selected.

2-Use the PRO1MICRO pen to take the sample from the surface to be analyzed. For this, follow directions described in Figure 1 (Also explained on PRO1MICRO's directions for use).

3-Once the solutions contained within the device have been correctly mixed, proceed with incubation (remember that the reading cone must not contain the swab). For this, insert the pen on the incubation/reading position (Figure 2). Make sure to place PRO1MICRO as it is indicated on the auto-reader label. When it is inserted in the correct position, it cannot be rotated (Figure 2).

4-Press and hold the **Pro** button for 2 seconds to initiate the incubation and reading process (15 minutes 60 °C). The lights and will start blinking.

Do not move or remove the pen from its position during the reading process, this can cause erroneous readings or even cancellation of the same. If the auto-reader detects that the protein pen has not been placed properly in the reading position, a ticket will be printed indicating: "Pen not detected. Check result visually". It is not possible to reuse an already incubated protein pen.

5-Once the incubation process is complete, an audible alarm indicates the end of the program. The sample is read and the protein value is calculated based on a BSA (Bovine Seric Albumin) calibration/standard curve. Then, the auto-reader will print a ticket confirming the calculated protein concentration. If the value is below the selected threshold, a green light will appear, indicating a **negative** result, if it is by above the selected threshold, a red light will appear, indicating a **positive** result. In both cases, one of the lights of the PRO panel will indicate the approximated

rank of concentration calculated between 1 to 5 µg; if the value exceeds the 5 µg all the lights on the panel they will light up. The threshold to determine a positive or negative result can be configured using the Bionova® traceability software. The configuration process is outlined in the Bionova® traceability software manual. The factory-set threshold is 1µg.

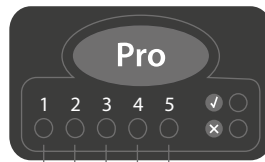
**NOTE:** 30 seconds before the end of the program, the auto-reader will emit an alarm that will last until the reading is complete, notifying the operator that the incubation time will be fulfilled. The color of the solution can be used for a qualitative reading only if the pen is removed once the program finalizes. If the pen is not removed at that time, the color of the solution will continue to evolve.

Read the directions for use of each product and be aware of the sensibility limits.


BIONOVA RAPID READOUT INCUBATOR SERIAL NUMBER: XXXX XXX PRO 1 PROGRAM: 15 min / 60°C DATE: DD/MM/AA TIME: HH:MM -10µg NEGATIVE PROTEIN: 0.0µg	IC10/20FR RAPID READOUT INCUBATOR SERIAL NUMBER: XXXX XXX PRO 1 PROGRAM: 15 min / 60°C DATE: DD/MM/AA TIME: HH:MM +1.0µg POSITIVE PROTEIN: 3.2µg	BIONOVA RAPID READOUT INCUBATOR SERIAL NUMBER: XXXX XXX PRO 1 PROGRAM: 15 min / 60°C DATE: DD/MM/AA TIME: HH:MM CANCELLED
---	--	--

**Quantitative Results:** For IC10/20FR-2.10 (March 2017) firmware version or higher (others versions could report different values).


1 µg 2 µg 3 µg 4 µg 5 µg  
Approximate amounts of BSA expressed in µg

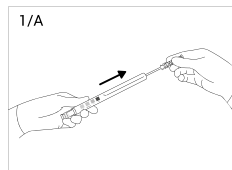Figure 1

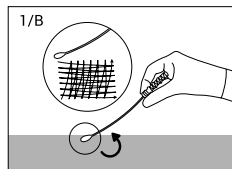
Remove the swab from the device. Carefully apply 2 drops of moisturizer on the swab.


Swab thoroughly the desired surface


# Instructions for use

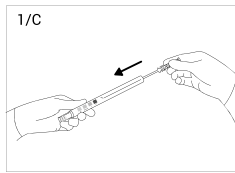
Place swab back into the device.

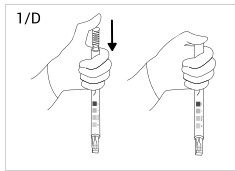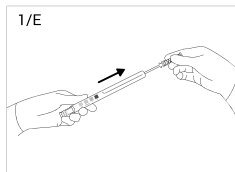
Place swab downwards firmly to activate.


Slide swab upwards.

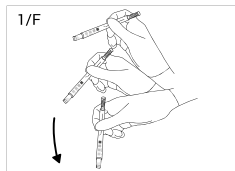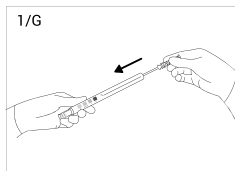
Shake vigorously and collect solution in the readout cone.


Slide swab downwards.

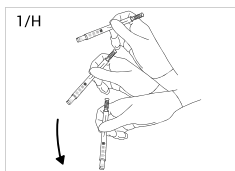
Shake for 15 seconds.

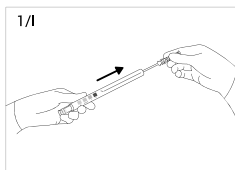
Slide swab upwards.

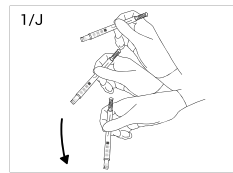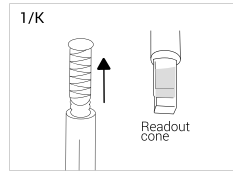Shake vigorously and collect solution in the readout cone.


Readout cone without swab.

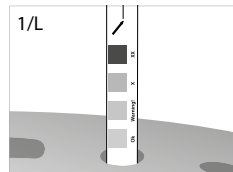Incubate and read.

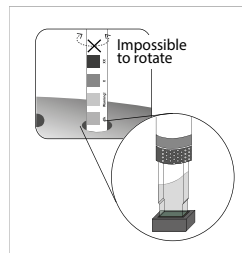**Figure 2**

Insert the pen on the incubation/reading position.

## Disposal

After reading, discard the hygiene indicators according to the sanitary regulations of your country. The hygiene indicators used cannot be reused (read the Instructions for use of Chemdye® PRO1 MICRO Hygiene Monitoring System).

## Audible alarm

The auto-reader is equipped with several alarm functions that indicate different states after the incubation. In the case of the biological indicators, an alarm will go off whenever the Bionova® IC10/20FR Auto-reader emits a positive result. This will allow the user to immediately detect a positive result without visually verifying the result on the equipment.

The alarm can be cancelled by pressing the button . The hygiene monitoring system, is equipped with an alarm that the auto-reader will set off 30 seconds before the program cycle ends and will last until

# Instructions for use

the reading is complete, notifying the operator that the incubation time will be fulfilled. The color of the solution can be used for a qualitative reading only if the pen is removed once the program finalizes. If the pen is not removed at that time, the color of the solution will continue to evolve.

## Canceling a reading

When a BI is removed from its position during incubation process, an audible alarm will sound to indicate the tube should be returned to its corresponding position.

If the indicator is not returned to the reading position within 10 seconds from its removal, the reading will be canceled and the auto-reader will print out a ticket confirming the cancellation.

In the hygiene monitoring system, the user can cancel the reading by pressing the button **Pro** of the chosen position and the button **⏸** for three seconds. The device will cancel the reading and the auto-reader will print out a ticket confirming the cancellation.

## Temperature monitoring

The Bionova® IC10/20FR Auto-reader contains an internal temperature control. If the temperature falls outside of the specified range ( $37\pm 2$  or  $60\pm 2$  °C), the blue temperature led light will start to twinkle.

Temperature can be externally monitored by using a Bionova® TB- IC1020 thermometer or other similar. The Auto-Reader IC10/20FR has a special hole for placing the thermometer which is located at the back of the device.

## Reprinting of results

The auto-reader allows reprinting of the last 12 results by fluorescence readout and the last result of hygiene monitoring reading. For printing the results, press the button **⏸** for 5 seconds.

## Remaining incubation time

The Bionova® IC10/20FR Auto-Reader also allows verifying the remaining incubation time in every reading position. Whenever a reading process is in course, press and hold the button **⏸** for five seconds, a ticket will then be printed containing the following information: reading position, incubation program and remaining incubation time.

```
BIONOVA      IC10/20FR
RAPID READOUT INCUBATOR
SERIAL NUMBER XXXX XXX
DATE: DD/MM/AA
TIME: HH:MM

PRO X
PROGRAM: X min / X°C
REMAINING TIME: HH:MM h

TUBE 1
PROGRAM: X min / X°C
REMAINING TIME: HH:MM h

TUBE 5
PROGRAM: X min / X°C
REMAINING TIME: HH:MM h
```

**NOTE:** This feature is available starting from the IC10/20FR Rev. 2.5 Firmware version.

For the detailed procedure for updating the firmware, please refer to the *Firmware Update* section of this user guide.

## Thermal paper specifications

Recommended paper: JUJO AF50KSE3 or similar (order code ICTP).

Paper width : 58 mm

Maximum paper thickness: 60 g/m<sup>2</sup>

Maximum diameter size: 50 mm

## Replacement of the paper roll

! The printer uses thermal paper rolls.

1. To replace the paper roll, pull the handle of the printer's door.

2. Open the printer's cover and remove the spent roll.

3. Place the new paper roll with the outer side up.

4. Close the printer cover by pressing on the sides of the lid.

# Instructions for use

## Care and cleaning

Clean the exterior surface of the Bionova® IC10/20FR Auto-reader with a damp cloth moistened with an small amount of detergent. Do not clean interior components. If additional cleaning of the wells is required, contact your distributor or manufacturer.

 Always unplug the IC10/20FR Auto-reader and allow it to cool before cleaning. Do not immerse the unit in liquid.

**NOTE:** Bionova® IC10/20FR Auto-reader does not need routine maintenance.

## Firmware update

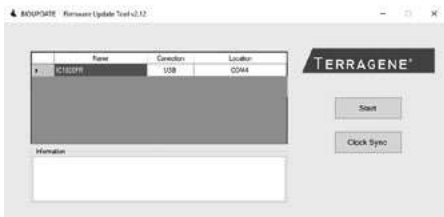
The IC10/20FR Auto-reader allows updating the firmware periodically (program that controls the device and defines its diferent functionalities) through the use of the Biupdate Software. This software verifies, by using the internet, the latest firmware version available for IC10/20FR Auto-reader; downloads it and installs it in a few seconds without the loss of any device data. The Biupdate Software must be downloaded, only once, from the following web site: <http://www.terragene.com.ar/productos/infection-control/incubators/software-downloads/?lang=en>.

Download file:  **Download Biupdate program**

### UPDATE PROCEDURE:

Before starting the process, make sure that the Bionova® software is closed. Turn off the auto-reader for five seconds, then turn it on and follow this steps:

- 1- Connect the auto-reader to a PC using a USB cable and make sure to have an internet connection.
- 2- Open Biupdate program.
- 3- Select the device and press Start button to update.

- 4- Wait until the software indicates that update process has been completed. The auto-reader will print a ticket indicating the new version of the Firmware.

**NOTE:** This process must be repeated every time that a new firmware version is available, this will be informed in the website.

## Calibration

The equipment calibration procedure is described in the attached software manual *Bionova® Traceability*

*Software for Automatic Reading and Traceability of Rapid, Super Rapid and Ultra Rapid Biological Indicators and Hygiene Monitoring System.* You can also download it from <http://www.terragene.com.ar/productos/infection-control/incubators/software-downloads/?lang=en>.

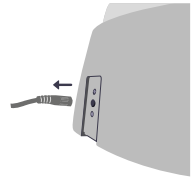
## Time zone change mode

This function allows you to advance or delay the internal auto-reader clock in order to adopt a different time zone.

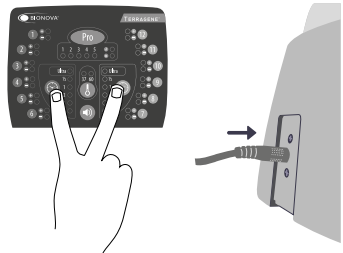
**NOTE:** Remember that it is possible to synchronize the time with the Biupdate® software and Bionova® traceability software.

### Enter Time zone change mode


De-energize the auto-reader.

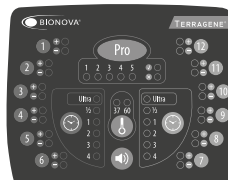Press and hold both  buttons simultaneously and energize the equipment. A beep will be emitted to indicate access to *Time zone change*.


**NOTE:** when entering *Time zone change* mode a green light will turn on in one of the incubation/reading position. This number corresponds to the current auto-reader time.


### Time zone change

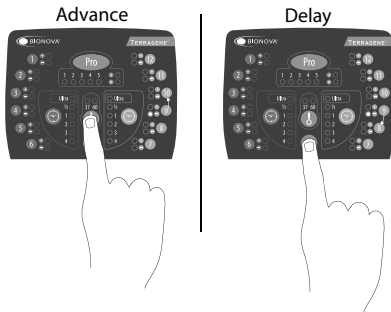To advance time, press  button as many times as hours you need to advance. To delay time press button  as many times as hours you need to delay.

# Instructions for use

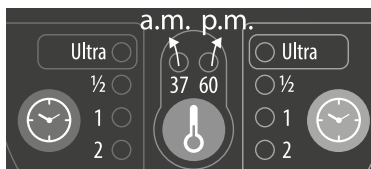
**NOTE:** 37°C light indicates a.m. 60°C light indicates p.m.

## Complete time zone change

To save changes press and hold **Pro** button (Figure 4) for three seconds. A beep will sound indicating that process has been completed successfully. A ticket with the new set time will be printed.

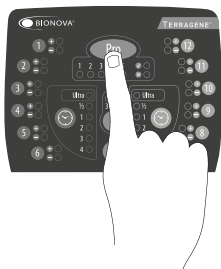**Figure 4**

## Cancel time zone change

To cancel time zone change process without saving the changes, disconnect power source.

## Time setting mode

Time setting functionality allows you establish a specific time regardless of auto-reader current time.

## Modification of the time using the Bioupdate® Software

Connect the auto-reader via the USB port to a PC and start-up the program. Select the Bionova® IC10/20FR Auto-reader and press the Clock Sync button, the auto-reader will be synchronized with the PC's clock. Synchronization can only be done as long as there is not an ongoing reading.

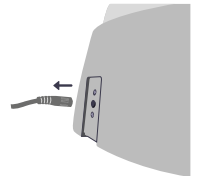
## Change the time using the Bionova® Traceability Software

Connect the auto-reader via the USB port to a PC and start-up the program. The Bionova® IC10/20FR Auto-reader will be synchronized with the PC's clock. Synchronization can only be done as long as there is not an ongoing reading.


## Change the time using the panel

### Enter Time setting mode

De-energize the auto-reader.

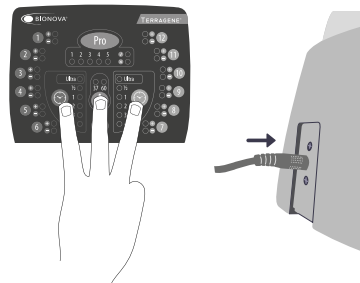Press and hold both **Pro** buttons and **A** button simultaneously and energize the equipment. A beep will be emitted to indicate access to *Time setting mode*.

## Hours setting


Lights corresponding to 1 to 10 incubation/reading positions indicate the digit value which is set. Lights from 1 to 9 indicate digit value from 1 to 9. Light 10 indicates 0 value.

In time setting mode, light 1 from PRO sector will turn on to indicate ten hour modification and incubation light corresponding to the 10th position will turn on to indicate setting started.

**NOTE:** 24-hour format.

! To **change** hours and minutes digits, use  $\downarrow$  and  $\uparrow$  buttons. To **increase** digit value press  $\downarrow$ ; to **decrease** press  $\uparrow$  (see Figure 5).

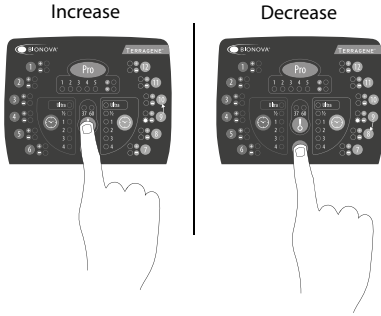**Figure 5**

**- Hours setting: Ten hours (Hh:mm)**

The blue light number 1 corresponding to PRO sector should be on. If any other light from PRO sector is on, press **Pro** button as many times as necessary to turn on the blue light number 1. This indicates ten hour setting.

To increase ten hours value, press  $\downarrow$  button. To reduce ten hours value, press  $\uparrow$  button. (See Figure 5).

**NOTE:** the possible values for ten hour are 0, 1 and 2.

**- Hours setting: Unit hours (hH:mm)**

Press **Pro** button to turn on the blue light number 2. This indicates unit hours setting. If any other light from PRO sector is on, press **Pro** button as many times as necessary to turn on the blue light number 2.

To increase unit hours value, press  $\downarrow$  button. To decrease unit hours value, press  $\uparrow$  button. (See Figure 5).

**Minutes setting**

**- Minutes setting: Ten minutes (hh:Mm)**

Press **Pro** button to turn on the blue light number 3. This indicates ten minutes setting. If any other light from PRO sector is on, press **Pro** button as many times as necessary to turn on the blue light number 3.

To increase ten minutes value, press  $\downarrow$  button. To decrease ten minutes value, press  $\uparrow$  button. (See Figure 5).

**NOTE:** possible values for ten minutes are 0, 1, 2, 3, 4 y 5.

**- Minutes setting: Unit minutes (hh:mM)**

Press **Pro** button to turn on the blue light number 4. This indicates unit minutes setting. If any other light from Pro sector is on, press **Pro** button as many times as necessary to turn on the blue light number 4.

To increase unit minutes value, press  $\downarrow$  button. To decrease unit minutes value, press  $\uparrow$  button. (See Figure 5).

**Finish hours setting**

To save changes press and hold **Pro** button (see Figure 4) for three seconds. A beep will sound, PRO green light will turn on and a ticket with the new set time will be printed to indicate the procedure has been completed successfully.

**NOTE:** If the modification has not been done correctly, the auto-reader will emit 3 beeps and the red light from PRO sector will turn on. Repeat the procedure.

**Cancel hours setting**

To cancel hours setting procedure without saving the changes, disconnect power source.

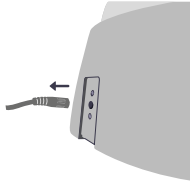
## Printing language change mode

This function allows you to change the auto-reader printing language.


**NOTE:** each language has a code number assigned:  
**1:** English | **2:** Spanish | **3:** Portuguese | **4:** Turkish

### Enter to *Printing language change mode*


De-energize the auto-reader.

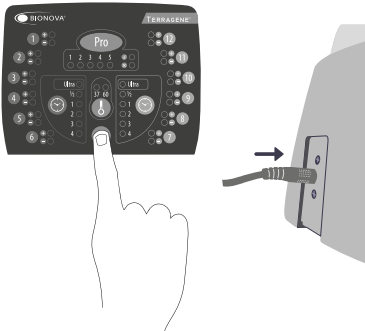Press and hold **Pro** button and energize the equipment. A sound will be emitted to indicate Access to *Printing language change mode*.

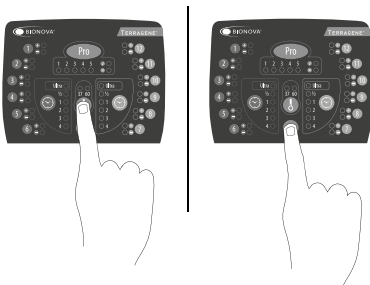Use **1** and **2** buttons to select the position number corresponding to language code.

## Finish printing language change

To save changes press and hold **Pro** button for three seconds. A sound will be emitted indicating that process has been completed successfully. A ticket with the new language will be printed.

# Troubleshooting chart

Fault	Possible cause	Action
<b>The auto-reader does not start</b>	Power source is not connected.	Check that the power source is connected to the power supply and that the plug is connected to the auto-reader.
<b>The auto-reader gives an error in one position during "Auto-test".</b>	A BI is placed in the position at the moment of starting the auto-reader.	Check that every position is empty at the moment of starting the auto-reader.
<b>The auto-reader gives an error in the "Autotest".</b>	Soil particles might be obstructing the light path between the sensors.	Ensure cleanliness of the incubation position. Use of air is recommended. Do not use solid objects that could damage internal components. Restart the auto-reader.
<b>The auto-reader does not run a BI reading.</b> The red light of that position is blinking.	The autotest gives an error in that position, which becomes disabled.	Make sure that the position is empty when starting the auto-reader. Restart the auto-reader.
<b>The auto-reader does not run a BI readout in any position.</b>	The incubation temperature is not stable.	Wait for temperature to be stable.
<b>The auto-reader does not run Chemdye® PRO1 MICRO readings.</b> The red light of that position is blinking.	The autotest gives an error in the PRO position, which becomes disabled.	Make sure that the PRO position is empty when starting the auto-reader. Restart the auto-reader.
<b>When initiating the PRO incubation program the machine makes a sound but it does not initiate.</b>	The incubation temperature is not stable.	Wait for temperature to be stable.
<b>The auto-reader does not allow to change the incubation program.</b>	Ongoing reading.	Wait for the readings to finish.
<b>The printer does not print and the printer indicating light blinks.</b>	The printer cover is not tightly locked.	Check that the cover is tightly closed.
	Printer without paper.	Place a new paper roll in the right direction.
<b>The printer releases unprinted paper.</b>	Paper roll is placed in the wrong direction.	Check the paper roll direction.
<b>The Bionova® traceability software does not detect the auto-reader.</b>	The auto-reader is turned off or the USB cable is disconnected.	Check that the auto-reader is turned on, the USB cable is connected and the COM port is correctly installed and accepted by the operative system.
<b>The update of the auto-reader fails.</b>	The auto-reader is connected to the Bionova® traceability software.	Close the program, restart the auto-reader and try again.

# Warranty

Terragene® S.A. guarantees both the quality of the product material components and the quality of its manufacturing process. Should any material or manufacturing faults be detected within the warranty period (1 year from the time of purchase), the only obligation of Terragene® S.A. will be product repairing or substitution.

## Limitation of liability

Terragene® S.A. shall not be liable for any loss or damage that may result from the unsuitable use of the equipment, negligence or user's full responsibility.

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<b>Constituent parts</b> .....	<b>3</b>
Description of use .....	3
References.....	4
Symbols .....	4
<b>Safety information</b> .....	<b>4</b>
Power supply specifications .....	5
Environment operating conditions .....	5
Serial/Batch Number.....	5
Record Table .....	5
<b>Instructions for use</b> .....	<b>6</b>
Start-up.....	6
Positive control.....	7
Interpretation of results .....	7
Disposal.....	7
Hygiene monitoring systems.....	7
Disposal.....	9
Audible alarm .....	9
Canceling a reading.....	10
Temperature monitoring .....	10
Reprinting of results .....	10
Remaining incubation time .....	10
Thermal paper specifications.....	10
Replacement of the paper roll.....	10
Care and cleaning .....	11
Firmware update .....	11
Calibration.....	11
Time zone change mode.....	11
Time setting mode.....	12
Printing language change mode.....	14
<b>Troubleshooting chart</b> .....	<b>15</b>
<b>Warranty</b> .....	<b>16</b>
Limitation of liability .....	16
Technical assistance .....	16

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Terragene S.A