# Bionova® Traceability Software

for Automatic Reading and Traceability of  
Rapid, Super Rapid and Ultra Rapid  
Biological Indicators and  
Hygiene Monitoring Systems

Version 3.2

*Innovation in technologies for Infection Control*

TERRAGENE®

Rev.8 | February 2019

# Introduction

The Bionova® Traceability Software for Automatic Reading and Traceability of Rapid, Super Rapid, Ultra Rapid Biological Indicators and Hygiene Monitoring Systems was designed to be used as a central tool for the traceability and recording of results of biological indicators with fluorescence readout. This comprehensive system allows linking all the information of the BI with the sterilizer and its manufacturer, the operator, cycle characteristics and all relevant data of the sterilization activities.

The system allows storing data, documenting it and processing it historically in a useful and flexible database.

The Hygiene Monitoring system allows tracing the status of the Protein Pen incubation and save all the measurements. This tool is operative from the Bionova® software version 2.2.

To download the Bionova® Software, follow the path <https://www.terragene.com.ar/en/products/infection-control/incubators/software-downloads/>.

[Click here to download Bionova® 3.2](#)

## Incubator connection

The Bionova® software version 3.0 has incorporated support for multiple incubators connected to a single computer via USB cable or connected to the same network with an Ethernet cable.

**NOTE:** Only one computer on the same network is allowed to run the Bionova® software version 3.0 (or later). Other configurations do not ensure the correct data collection.

Below, there are some recommendations for an optimal communication between a USB incubator and the computer:

- Always use high quality USB cable, whose length is no greater than 150 centimetres.
- Connect the incubator to the computer directly, avoiding the use of adapters or USB HUB.
- Ensure that the USB ports remain connected firmly at both ends.

For the connection of Ethernet incubators there are also some recommendations for an optimal communication:

- Always use high quality Ethernet cable, category 5 or higher.

- Avoid using WiFi connections, unless necessary.
- Ensure that the Incubator has a valid IP address.
- The computer firewall should not block the communication of the program or the “Incubator Finder” Windows Service.

## First execution

When executing the program for the first time, the user will be requested to set a password for the Admin account. This account corresponds to the application administrator user, who can enable additional accounts for other users.

Enable administrator account

Username: admin

Password: [masked]

Repeat password: [masked]

Good

OK Close

**!** A strong password contains at least 5 characters length, upper and lower case, numbers and special characters (!?%#).

After having defined the password for the Admin account, the Admin user will be requested to complete the options corresponding to the general configuration of the program:

Bionova - Options

General

Company / Hospital: Terragene

Department: [empty]

Address: [empty]

Section: [empty]

Select the threshold level (Pst):  $\geq 1.0$  (Ug)

Example

Protein Level:  $\geq 1$  ug Result: Positive

Protein Level:  $< 1$  ug Result: Negative

Apply Close

# Introduction

This information is very important since it will be shown in the headline of all reports. Once the configuration is completed, the Admin user may start operating the program. This configuration can be changed later by using the option: [Bionova >> Options](#)

## Starting the application

The following actions take place every time the user starts up the application:

**1- Presentation screen:** this screen is the first message displayed when the application is launched. The presentation screen disappears automatically in 10 seconds, but the user has the possibility to skip it by clicking on the "Close" button, which appears in the upper right corner.

**2- Access control:** the last step in the sequence of start up is the request of access credentials to operate the program. Here the user must enter the username and password, whereupon can start using the application.

## Access rights and users commutation

The Administrator user (Admin) has full access rights to operate the application. By using this user account, the Administrator will be able to:

- Add or eliminate other user accounts (see *User account management*).
- Reset passwords for existing accounts (see *Password reset and modification*).
- Modify the application settings.
- Display the history of results in full.

Additional user accounts (corresponding to the operators) can only see the results of the analysis that

they have done, but not those that have been made by other users.

In case that different user accounts have been enabled, it is possible to switch from one user to another by using the option [Bionova >> Sign out](#). When logging out, the program presents a screen asking for a user name and password, and from there you can re-enter the program with a different account.

**NOTE:** Bionova® Software can only be used with incubators that include 2.4 or higher Firmware version.

**NOTE:** In case the Admin user forgets the password, please contact [info@terragene.com.ar](mailto:info@terragene.com.ar) for password clearance.

## User account management

- With the option [Bionova >> Create a new user account](#) the Admin user can add a new user account to the program.

- Through the option [Bionova >> Users management](#) the Admin user can deactivate existing accounts and reset passwords.

## Password reset and modification

The Admin user can reset the password of other users; this feature can be useful in case a user has forgotten their password. Once the administrator has reset the password of a user, the user may enter the program using as password their user name followed by a hyphen and the numbers 321. For example, if the user account is called Bionova and administrator resets their password, the new password will be Bionova-321. Moreover, all users have the possibility to change their own password, using the option:

[Bionova >> Change password](#)


# Language configuration

## Language selection

The application is available in five languages:

- **English**
- **Spanish**
- **Portuguese**
- **Turkish**
- **Chinese**

When installing the program, it is possible to select the language you want to use during installation.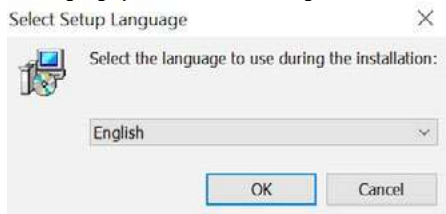

There are three important points to consider:

1. The language you choose to install the program refers exclusively to the language to be used during the installation process.
2. The actual implementation is always installed in all available languages (regardless of the language you have chosen when installing).
3. When the program starts up, the application will automatically be presented in the appropriate language according to the language of the Operating System:
  - If running on an Operating System in Spanish, the program will be displayed in Spanish.
  - If running on an Operating System in Portuguese, the program will be displayed in Portuguese.
  - If running on an Operating System in English (or in any other language other than those supported), the application will be displayed in English.

## How to force the language selection


As mentioned above, the application is always automatically shown in the corresponding language according to the language of the operating system. However, it is also possible to force the use of a specific language, different from the one of the Operating System.

In that case, these steps should be followed:

1. Locate the shortcut icon to the Bionova® program.

This icon can be on the desktop, or in the Applications menu, depending on which options are selected when installing the program.

2. Click the right mouse button on the program's icon and select the option *Properties*.
3. When selecting the option *Properties*, a window like the one shown below, containing the configuration of the shortcut will be displayed: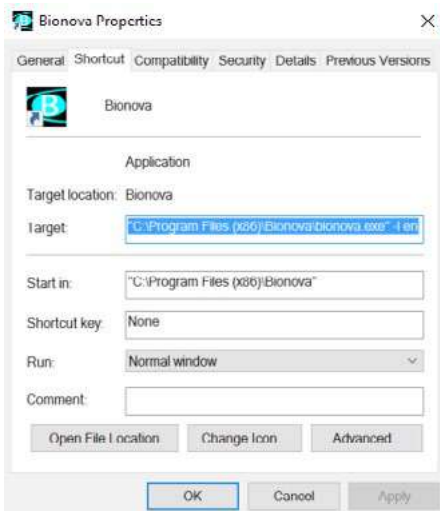

In this window, in the Target box, you must add:

-l es (**space hyphen** followed by the letter **l** in **small letter**, a **space** and the word **es**).

**NOTE:** the word **es** refers to the language Spanish and tells the program you want to work in that language. To force the use of other language: English: **en**, Portuguese: **pt**, Chinese: **ch** and Turkish: **tr**.

4. Finally, press the Apply button.

Once you have modified the configuration of the shortcut, next time you run the application, it will be displayed in the selected language (regardless of the Operating System language).

## Main screen

The main screen of the application is organized into four tabs:

**1. Biological Indicators Tab:** This tab shows the data of the ongoing analysis, using automatic information provided by the incubator. Moreover, from this section the operator can complete all the data that should be uploaded manually.

**2. Results Tab:** This tab shows the stored results, corresponding to prior analysis. In this section the complete history of results can be consulted, allowing organizing and filtering the results in different ways.

**3. PRO Tab:** This tab shows the ongoing PRO analysis, giving the possibility to check the remaining time, complete all the relevant information and save the result once finished.

**4. PRO Results Tab:** This tab shows all the saved results of protein analysis, allowing to order and filter the results.

In addition to these tabs, the main screen contains the following elements for the interaction with the program:

- **Title bar:** Displays the application name and the name of the user who is running the program (in brackets).

- **Main menu:** Allows accessing the various functions provided by the application. Main functions are available within Bionova® menu.

- **Toolbar:** The toolbar is located under the main menu and is composed of a set of buttons that allow a quick access to certain functions in the program such as results record, print, etc.

**NOTE:** Some buttons are available only when the user is working on the Biological Indicator tab, while others only work when using the Results tab. The same is true for the PRO and PRO Results Tab.

- **Status bar:** Is the bar that appears at the bottom of the screen, containing information on the operation of the program (in *Status bar* section you will find more details about the information displayed).

### Status bar

At the bottom of the main screen there is a status bar divided into three sections:

- **Connection to the incubator:** it shows the connection status with the incubator, which can be "Connected" (highlighted in green) or "Disconnected"

(highlighted in magenta). When the connection with the incubator is established, it shall also indicate the incubator model, lot and serial number.

- **Selected program:** it shows which of the programs have been selected in the incubator, followed by the current temperature (separated with a slash). When no program is selected in the incubator, it will show "Stand-by".


- **Last reading data:** it shows the date and time of the last reading data. It also indicates the actual temperature detected by the system. Another data is shown as "S", and the value looks like a fraction or ratio. This value helps to confirm if the selected temperature has been correctly reached. The numerator can take the values 1 or 2, where 1 means "unstable" and 2 means "stable". The denominator corresponds to the selected temperature (37 or 60 °C). The item "Readings", displays the number of times that the software has been communicated with the incubator.

### Incubator selection

At the toolbar there is a drop-down list that shows all the incubators connected to the computer or the local network. In order to see analysis in progress, archive data or perform temperature calibration, an incubator must be selected from the list.

While no incubator is selected, the analyses are tracked on the background but no data will be archived. If an analysis is done on the background, a notification will appear informing in which incubator the event occurred.

# Biological Indicators Tab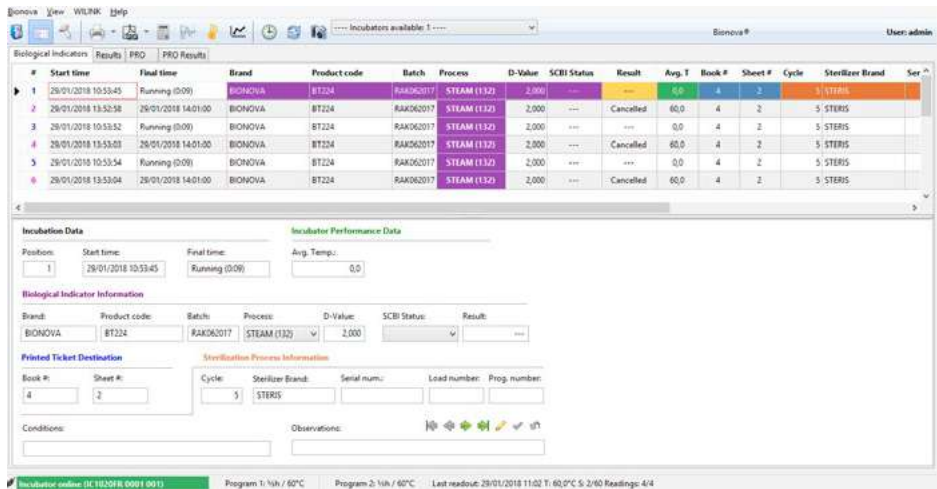

The Biological Indicators Tab contains a grid similar to a spreadsheet, where the user can view the data for all the ongoing Biological Indicator (BI) readings. The grid comprises 12 rows, each of which corresponds to a position in the incubator. In addition, the grid displays a series of columns that will guide the user to complete all the relevant information about a specific BI result before saving it. Some of these fields will be automatically filled with information provided by the incubator, while other fields should be manually completed (details given below for each column).

Next, the user will find the list of fields (columns) with a short guide for filling with the correct information.

- **#:** (Automatic information) Position of the incubator where the BI is being incubated and read.
- **Start time:** (Automatic information) Gives the exact date and time of beginning of the incubation.
- **Final time:** (Automatic information) Gives the exact date and time of finalization of readout. During incubation, this field will display the elapsed incubation time.
- **Purple fields:** Information about BI
- **Brand:** (Manual information) Brand of the BI.
- **Product code:** (Manual information) Product code of the BI.
- **Batch:** (Manual information) Batch number of the BI.

- **Process:** (Manual information) Conditions for calculation of the D-Value. The user can find this information in the product Directions for use (which includes the Quality Certificate of that BI batch).

- **D-Value:** (Manual information) Provides information about the resistance of that BI batch. The user can find this information in the product Directions for use (which includes the Quality Certificate of that BI batch). Be careful to choose the correct D-Value considering the "Process" selected in the previous field.

- **SCBI Status:** (Manual information) Dropdown menu. You have to choose between 2 options: "Positive" (positive control, non-exposed) or "Exposed" (to the sterilization cycle).

• **Yellow fields:** Result: (Automatic information) Actual result given by the incubator (Positive, Negative or Cancelled).

• **Green fields:** Avg. T: (Automatic information) Average temperature during the incubation period of that BI.

• **Blue fields:** Logbook information (book where you keep your records, including the incubator's printed ticket)

- **Book #:** (Manual information) Complete with the logbook number corresponding to sterilization process.

- **Sheet #:** (Manual information) Complete with the

## Biological Indicators Tab

logbook page number corresponding to sterilization process.

- **Orange fields:** Sterilizer and cycle information

- Cycle:** (Manual information) Cycle number, provided in the sterilizer screen and/or ticket.

- Machine brand:** (Manual information) Brand of the sterilizer.

- Serial num.:** (Manual information) Serial number of the sterilizer, in case you have 2 or more of the same brand.

- Load #:** (Manual information) Load number of the day.

- Prog. #:** (Manual information) Program of the sterilizer used for that cycle.

- **Conditions:** (Manual information) You can fill this box with details about the cycle (not mandatory information).

- **Observations:** (Manual information) You can fill this box with any information you consider relevant and it is not already considered in the other fields (not mandatory information). This is the only field that can be filled or modified after the result has been saved in the Result Tab.

To modify the data of a field, click the corresponding box and start typing.

The first time you enter a new input, you have to type it down. Then, next time you choose a box under the same title (or column), it will display a dropdown menu so that you only have to choose the desired option (no typing needed).

**NOTE:** This is true only for alphanumeric inputs; for numerical inputs you have to type down the value every time.

### Color codes

The first column in the Biological Indicators grid (#) presents the numbers corresponding to each position of the incubator. The numbers of that column may adopt different colors:

- **Blue:** This color is used to indicate the positions in which there are tubes in process of incubation.

- **Purple:** This color indicates tubes whose analysis are completed.


- **Green:** Indicates that the analysis of the tube in that position has finalized, but the incubator already possesses data of a new analysis initiated in the same position. In these cases, the program cannot import the

new data until the user releases the position, archiving the results permanently. The highlight in green of those cases is intended to alert the user that they must archive those results to release the position on the Biological Indicator grid. Once the position is released, the program will import the data that matches the new analysis.

### Expedited data upload

In order to simplify and expedite the data uploading, the program provides a records copy mechanism.

This mechanism works as follows. Suppose we are analyzing 10 samples whose general data are coincident (brand, product code, lot, sterilizing machine, sterilizing cycle, etc.). What we do in that case is to load the data for the first sample. Then we have to right-click on the incubation position and select the "Copy data from" option. This allows uploading repetitive data rapidly and with minimal effort.

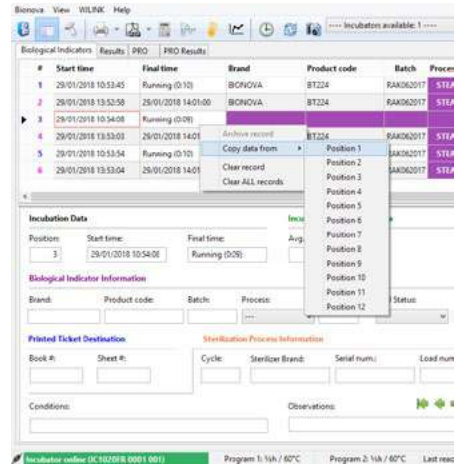**NOTE:** the contextual menu is accessed by pressing the right mouse button on the grid.

# Biological Indicators Tab

## Biological Indicators barcode reader

The program has an application through which the barcodes that encode BIs manufacturing information can be scanned, enabling the detection of their brand, the product code, the batch it belongs to, and the expiration date.

For using the data upload application with the reader, select the position you wish to complete and select the icon .


Use the product's box to read the barcode. If the reading is completed, apply the data to the BI position. The system works with EAN 128 codes in which the

batch and expiration date of the product is indicated. In those products in which the information is separated into two different codes, repeat the reading on each.

This tool allows you to choose which rows you want to apply the barcode provided information to. For example, if you have selected position number 5 when pressing the barcode button, the system will give you the next options for applying the information:

- Selected test (position 5)
- All running tests
- All tests

Then, you choose the desired option and press the Apply button. After that, press the Close button and you will see the barcode information given in the grid.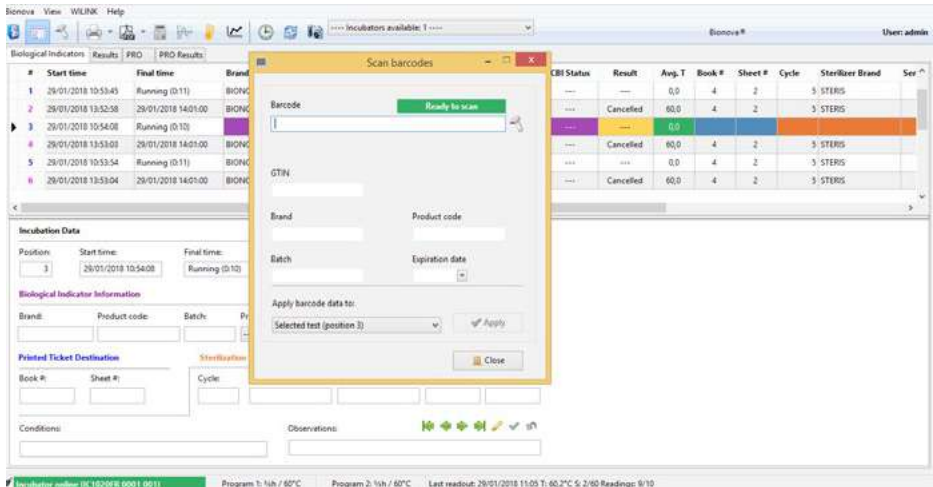

# Biological Indicators Tab

## Form view

In addition to the grid described above, the Biological Indicator tab also offers a view that shows the same data as in the grid but presented as a form.


The form is divided into several sections (Incubation Data, Machine Performance Data, etc), grouped with the same Color code as in the grid.

In the lower right of the form, on the box "Observations" there is a set of buttons that allow the user to move from one record to another, as well as apply or cancel the changes made to the current record.

It is worth mentioning that working with the grid or the form is indistinct, since both allow you to query and modify the data corresponding to the ongoing analysis. If you use the grid, the information will be automatically loaded in the form and vice versa.

**NOTE:** Users who prefer working exclusively with the grid can hide the form view using the option:

View >> Show or hide form view >> or by pressing the corresponding button on the toolbar.

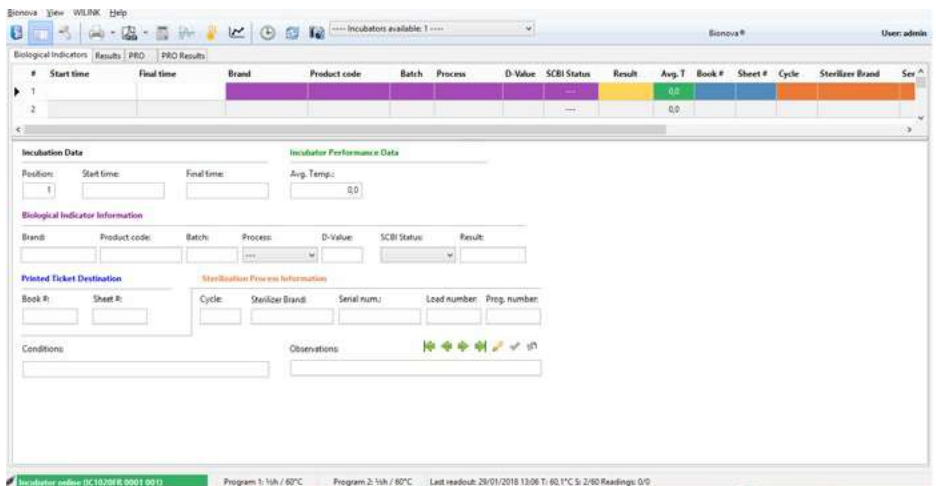# Biological Indicators Tab

## BI's results storage

When storing a set of results, these will be transferred from the Biological Indicator tab to the Results tab, remaining stored and available for future reference. Before performing permanent storage of data, the program performs a number of verifications aimed at ensuring the completeness thereof. This guarantees that the user won't miss any important information about that test when saving the BI result.

The data listed below is mandatory, so the operator must complete it before being able to archive the results:

- **All purple fields:** Brand, Product Code, Batch, Process, D-Value, SCBI Status
- **Both blue fields:** Book # and Sheet #
- **Orange fields:** Cycle, Machine brand, Serial num., Prog. # (all but Load #).

## Automatic storage

After BI's incubations end, the results are automatically transferred to the grid of the Results tab. It is worth mentioning that the storage of the results will be automatically performed only when the mandatory data of the BI test is complete at the time of finishing the analysis.

If the analysis of a BI finalizes and its information is not complete, the user must complete the missing information (on the Biological Indicator tab) and then save the results manually. This is explained in the following section.

## Manual storage

### • Storing an individual result

To permanently store the information corresponding to a particular BI, we have to right-click on the incubation position and select "Archive Record".

### • Storing a set of results

To permanently store two or more results in a single step, the user can use the button "Save results", available in the toolbar:

## Results Tab

The Results tab contains a table displaying all the loaded information that has been completed during incubation of the BIs and correspondingly saved after incubation has finished.

This grid is quite similar to the one in the Biological Indicator tab, but contains some additional data:

- **Lot:** (Automatic information) Lot number of the incubator, information provided by the incubator itself.
- **Serial #:** (Automatic information) Serial number of the incubator, information provided by the incubator itself.
- **Operator:** (Automatic information) Name of user that was logged in during incubation of this BI. Given by the software.
- **Visual result:** (Manual information) Dropdown menu where you can choose between “Positive” or

“Negative”. You need to complete this field only if you have performed the optional incubation for 48 hours (visual confirmation through culture medium color change). Please, read the BI’s Directions for use before performing this optional incubation.

- **SPR (Sterilization Process Risk):** (Optional information) This value corresponds to an optional calculation. For further information, please refer to the *Calculation of the Sterilization Process Risk (SPR)* section. The data displayed in the results grid cannot be modified, with the exception of the fields “Visual Result” and “Observations”, which do allow making changes. The SPR value can only be registered and calculated by using the corresponding tool (refer to the “Calculation of the Sterilization Process Risk (SPR)” section).

#	Lot	Serial #	Operator	Start time	Final time	Program	Brand	Product code	Batch	Process	D-Value	SCBI Status	Result
9	IC1020FR001	1	admin	26/01/2018 13:11:00	26/01/2018 13:31:00	1hA / 60°C	Z	2	2	STEAM (1326)	2,000	Positive	Negative
10	IC1020FR001	1	admin	26/01/2018 13:11:00	26/01/2018 13:31:00	1hA / 60°C	Z	2	2	STEAM (1326)	2,000	Positive	Negative
11	IC1020FR001	1	admin	26/01/2018 13:11:00	26/01/2018 13:31:00	1hA / 60°C	Z	2	2	STEAM (1326)	2,000	Positive	Negative
12	IC1020FR001	1	admin	26/01/2018 13:11:00	26/01/2018 13:31:00	1hA / 60°C	Z	2	2	STEAM (1326)	2,000	Positive	Negative
2	IC1020FR001	1	admin	29/01/2018 13:52:58	29/01/2018 14:01:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
4	IC1020FR001	1	admin	29/01/2018 13:53:03	29/01/2018 14:01:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
6	IC1020FR001	1	admin	29/01/2018 13:53:04	29/01/2018 14:01:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
8	IC1020FR001	1	admin	29/01/2018 13:53:13	29/01/2018 14:01:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
11	IC1020FR001	1	admin	29/01/2018 13:53:06	29/01/2018 14:01:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
12	IC1020FR001	1	admin	29/01/2018 13:53:05	29/01/2018 14:01:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
1	IC1020FR001	1	admin	29/01/2018 13:52:13	29/01/2018 14:06:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
3	IC1020FR001	1	admin	29/01/2018 13:52:20	29/01/2018 14:06:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
5	IC1020FR001	1	admin	29/01/2018 13:52:30	29/01/2018 14:06:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
7	IC1020FR001	1	admin	29/01/2018 13:56:35	29/01/2018 14:06:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
9	IC1020FR001	1	admin	29/01/2018 13:56:30	29/01/2018 14:06:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled
10	IC1020FR001	1	admin	29/01/2018 13:56:28	29/01/2018 14:06:00	1hA / 60°C	BIONOVA	BT224	RAK062017	STEAM (1326)	2,000	Positive	Cancelled

### Sorting and filtering of results

At the top of the Results tab there is a panel that offers different options to filter or sort the results shown. To sort the results use the selector “Sort by”, which appears in the upper left corner, and then choose the desired criteria. Otherwise, you can directly click in the field’s title and automatically sort by that criteria.

The rest of the boxes allow specifying different criteria for filtering. Once the criteria is defined, the filter can be applied by pressing the button located at the right end of the panel ( button allows access to more options for filtering data).

The following example shows the application of a filter to display the analysis performed from 22/01/2018, considering only the positive results.

## Results Tab

Showing results matching the given criteria

#	Lot	Serial #	Operator	Start time	Final time	Program	Brand	Product code	Batch	Process	D-Value	SCBI Status	Result
1	IC102PR001		1: admin	26/01/2018 15:11:52	26/01/2018 15:19:00	1/h / 60°C	2	2	2	STEAM (113Z)	2,000	Positive	Positive

Incubator online IC102PR001 0011    Program 1: 1/h / 60°C    Program 2: 1/h / 60°C    Last readout: 26/01/2018 12:11 T: 60.0°C S: 2:60 Readings: 152/162

### Sterilization Process Risk (SPR) calculation

The application allows calculating the Sterilization Process Risk (SPR) of a set of results. This function can be accessed by the  button in the toolbar. This button only works when the user is viewing the BI Results tab.

The idea of this tool is to take advantage of the BI's results to monitor not only the sterility of a certain cycle but also the performance of a specific sterilizer in time (historical performance). This calculation relies in a very sophisticated and exclusive algorithm that takes into consideration the next information: cycle number, sterilizer (brand and serial number), number of processed BIs (in case you use more than one BI in a given cycle), resistance of the BI batch (through the informed D-Value), results of BIs (positive or negative) and the time to give positive result (in case of having a positive result), among others.

The SPR calculation will be informed as None, Low, Medium or High, according to applicable international regulations. Nevertheless, the user must consider that a Low risk doesn't mean that the cycle is not risky; a positive result implies that the cycle being monitored cannot guarantee sterility of the load, so the load needs to be reprocessed, without exception. For this reason, the SPR value is not intended to be examined cycle by cycle but, on the contrary, considering a group of cycles during a considerable period of time. For this reason, these SPR calculations are used to represent the sterilizer performance by using the Historical Sterilizer Performance tool (see next section). By implementing this innovative tool, the user will be able to perform a thorough inspection of every sterilizer in the CSSD by saving a lot of time for the audit sterilizer's examination. In this way the SPR provides useful data that can be used to reduce productive costs, foresee the sterilizer repairs, or revise the sterilization procedures.


----- Incubators available: 0 -----

## Results Tab

When pressing the  button, the program will display a form where the user must indicate the grouping criteria for the calculation (date of incubation, sterilization cycle, sterilizing machine and number thereof). Once the criteria are defined, the user must press the button "Compute SPR" for the program to perform the calculation.

After completing the calculation, the program will show the result. Pressing the "Apply" button, the SPR will be transferred to the results grid (where it will remain available for reference in the "SPR" column).

**NOTE:** This tool should be validated by each institution according to the specific requirements for each application.

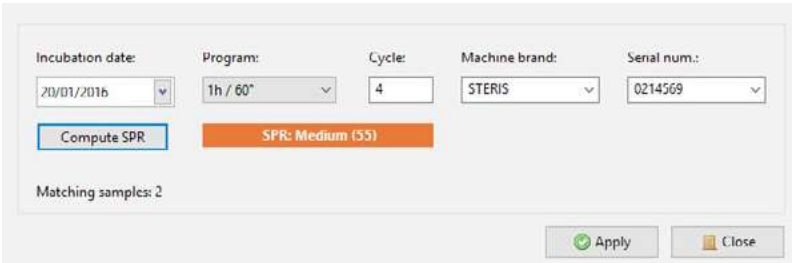The screenshot shows a web-based form for calculating the Sterilization Process Ratio (SPR). The form includes five input fields: "Incubation date:" with a dropdown menu showing "20/01/2016"; "Program:" with a dropdown menu showing "1h / 60°"; "Cycle:" with a text input field containing "4"; "Machine brand:" with a dropdown menu showing "STERIS"; and "Serial num.:" with a dropdown menu showing "0214569". Below these fields are two buttons: a blue "Compute SPR" button and an orange "SPR: Medium (53)" button. Underneath the buttons, it says "Matching samples: 2". At the bottom right of the form are two buttons: a green "Apply" button and a grey "Close" button.

**NOTE:** To simplify the specification of the criteria for grouping, when entering the form for the calculation of the SPR, the program completes the boxes with the values corresponding to the result that has been selected at the time of entry.

## Historical Sterilizer Performance


By implementing this innovative tool, the user will be able to perform a thorough inspection and audit of every sterilizer in the CSSD. In this way the SPR provides useful data that can be used to reduce productive costs, foresee the sterilizer repairs, or revise the sterilization procedures.

This tool will only be active when working on the Results tab (graph button next to the SPR button)

## Results Tab

The Historical Sterilizer Performance consists of a chart that represents all the SPR values for every cycle. When pressing the  button, a window will appear as below.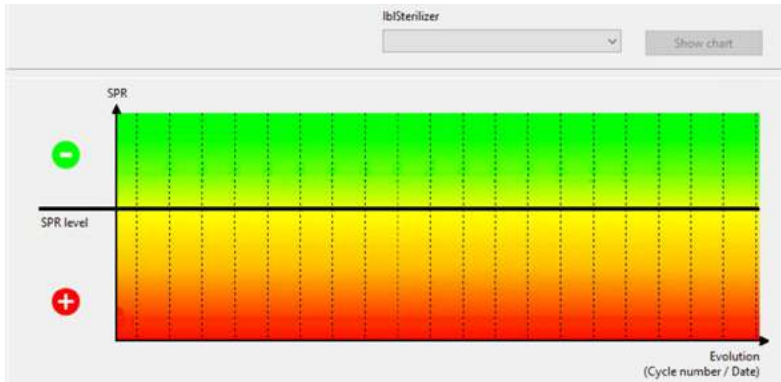


In the X axis the performance Evolution is represented, through the Cycle number/Date, while in the Y axis the SPR level is drawn. You choose the sterilizer you want to analyze through the dropdown menu and press the "Show chart" button. This way, every sterilizer will have its own historical performance.

The chart is divided into 2 halves; the top one corresponding to Negative SPR values (green zone of the chart) while the lower half (orange and red zones) correspond to Positive SPR values (including Low, Medium and High). Every cycle will have a SPR value represented by a dot.

Next, a few examples of sterilizer performance evolution and their interpretation to illustrate the versatility and usefulness of this tool.


## Results Tab

### Example 1. Correct sterilizer performance.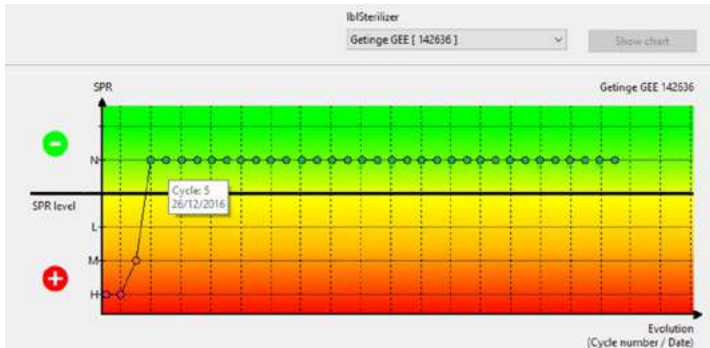

In this case, the user can see that the sterilizer was showing some failure, as shown by the first three SPR values with High and Medium levels. In this case, thanks to the use of this tool it was possible to detect this sterilizer malfunction and arrange for a proper maintenance. The user can corroborate through the chart that after this maintenance the sterilizer performance was acceptable since the SPR values return to Negative levels (Cycle 5 on the 26th of December 2016), corroborating the correct sterilizer performance from that moment on.

### Example 2. Random sterilizer failure.

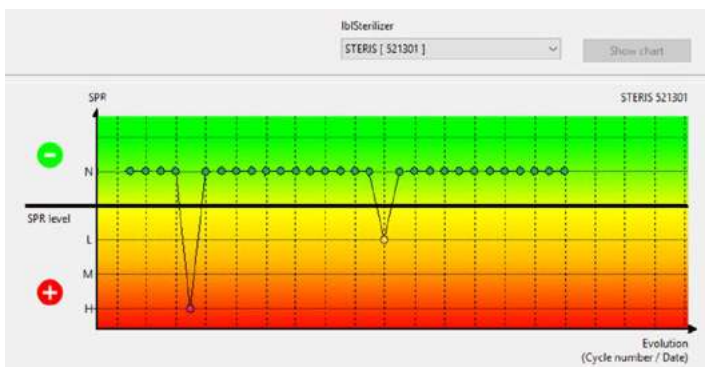
The sporadic occurrence of positive SPR values is an indication of random failure, which could be due to premature cycle abortion, operator mistake, wrong BI positioning inside a test pack or sterilizer chamber, overloading of the sterilizer chamber, among others. Detecting this type of failures could definitely help improve the general protocol of the sterilization process by analyzing the reasons of these failures whenever they are identified. If the user positions the mouse pointer in any dot, both cycle number and date will be shown, helping to identify the exact cycle. With this information and the Results grid filtering tool, the user can trace the rest of information in order to evaluate the possible causes and revert the situation.

## Results Tab

### Example 3. Sterilizer malfunction.

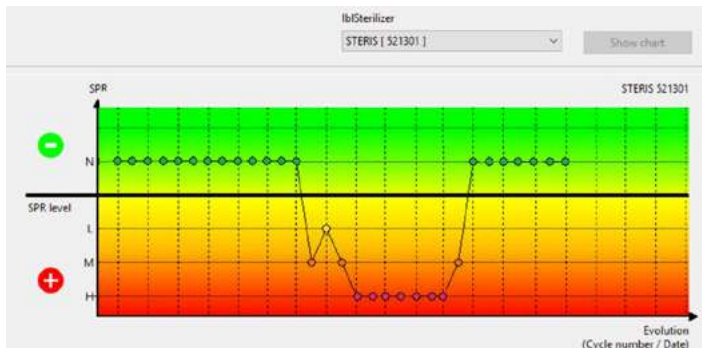This is an example of sterilizer failure chart, where you can observe that the sterilizer consecutively shows positive SPR levels, which demonstrates that there are no random situations but a consistent failure in some of the sterilizer's parameters. Here, the sterilizer was showing normal Negative SPR values and started to develop positive values, until the sterilizer was repaired and SPR values return to negative levels. If you obtain a chart like this, you need to urgently stop using that sterilizer, program its maintenance/repair and re-evaluate its performance after repair (where you need to obtain consecutive negative SPR values).

## PRO Tab

The PRO Tab contains a form in which all relevant information about protein analysis is displayed, along with the incubation remaining time.

This form shows a number of editable fields, very similar to those found in the Biological Indicator Tab, such as:

- **Incubation Data:** Information about the incubation process.

- Date:** (Automatic information) Displays date and time of Protein pen incubation.

- Remaining time status bar:** (Automatic information) Displays the incubation remaining time.

- **Operating Data of the Incubator.**

- Avg. Temp.:** (Automatic information) Average temperature during the incubation period of that Protein test.

- **Hygiene Indicator Data:** Information about the Protein test used.

- Product code:** (Manual information) Product code of the protein test.

- Brand:** (Manual information) Brand of the protein test.

- Batch:** (Manual information) Batch number of the protein test.

- Monitoring type:** (Automatic information) Indicates that the monitoring type being used complies with the HTM01-01.

- Result:** (Automatic information) Protein test result according to the incubator's information. This result can be "Positive" or "Negative".

- Visual result:** (Manual information) Protein test visual confirmation through color change interpretation.

- Protein level (µg):** (Automatic information) Quantitative Protein test result according to the incubator's information. This result is given in micrograms (µg) of protein. This information is provided through a BSA calibration curve.

- Threshold:** Defines the protein level (µg) from which the hygiene indicator result will be considered as positive. For configuration enter [Bionova >> Options >](#). Reference default value: 1 µg.

- **Printed Ticket Destination:** Book where you keep your washing records

- Book #:** (Optional information) Complete with the logbook number corresponding to the washing process.

- Sheet #:** (Optional information) Complete with the logbook page number corresponding to the washing process.

- **Process Information:** Information about the washing cycle and washer machine.

- Cycle:** (Manual information) Cycle number, provided

in the washer ticket or display.


- Prog. Number:** (Manual information) Program used for that specific washing cycle.

- Surface:** (Manual information) Gives information about the item that has been sampled.

- Identification:** (Manual information) Descriptive name of the washer machine used for the process. This field must be filled utilizing the Search Washer tool .

- Washer brand:** (Manual information) Washer's brand. This field must be filled utilizing the Search Washer tool .

- Machine serial:** (Manual information) Serial number of the washer. This field must be filled utilizing the Search Washer tool .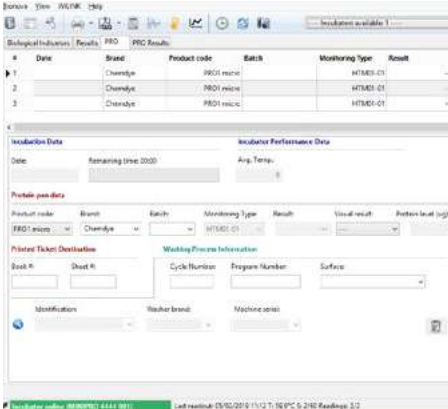

The screenshot shows the Bionova PRO Tab software interface. At the top, there's a navigation bar with 'Biological Indicators', 'Results', and 'PRO Results'. Below this is a table with columns for Date, Brand, Product code, Batch, Monitoring Type, and Result. The table contains three rows of data. Below the table, there are several sections for data entry: 'Incubation Data' with fields for Date, Remaining time (20:00), and Avg. Temp.; 'Protein pen data' with fields for Product code, Brand, Batch, Monitoring Type, Result, Visual result, and Protein level (µg); 'Printed Ticket Destination' with fields for Book # and Sheet #; 'Washing Process Information' with fields for Cycle Number, Program Number, and Surface; and 'Identification' with fields for Washer brand and Machine serial. At the bottom, there's a status bar showing 'Incubator code: 88888888 6144 001' and 'Last reading: 05/05/2018 11:03:11 16 °C 5.240 Readings 3/2'.

## Storing PRO Results

Once the Protein test incubation has finished, the amount of protein will be shown in the "Protein Level" field, and two buttons will be automatically enabled in the lower right corner of the form. The left "bin" button will allow the user to eliminate that reading and the right one will let the user save the Protein test result. If you want to save the results you have to complete all the mandatory information, otherwise, you won't be able to permanently archive the protein tests results. Mandatory fields include: product Brand and Batch, Visual result, Cycle, Surface, Washer Identification, Brand and Serial.

The screenshot displays the Bionova PRO software interface. At the top, there is a menu bar with 'Bionova', 'View', 'WILINK', and 'Help'. Below the menu is a toolbar with various icons. A status bar indicates 'Incubators available: 1' and 'User: admin'.

The main window shows a table titled 'PRO - PRO Results' with the following columns: #, Date, Brand, Product code, Batch, Monitoring Type, Result, Visual result, Protein level (ug), Avg. Temp., Book #, Sheet #, and Cycle. The table contains three rows of data:

#	Date	Brand	Product code	Batch	Monitoring Type	Result	Visual result	Protein level (ug)	Avg. Temp.	Book #	Sheet #	Cycle
1	05/02/2019 11:13:00	Chemdye	PRO1 micro		HTM01-01	Cancelled	---	0	59.9			
2		Chemdye	PRO1 micro		HTM01-01	---	---	0	0			
3		Chemdye	PRO1 micro		HTM01-01	---	---	0	0			

Below the table, there are several sections for data entry:

- Incubation Data:** Includes fields for Date (05/02/2019 11:13:00) and Remaining time (00:00).
- Incubator Performance Data:** Includes a field for Avg. Temp. (59.9).
- Protein pen data:** Includes dropdown menus for Product code (PRO1 micro), Brand (Chemdye), Batch, Monitoring Type (HTM01-01), Result (Cancelled), Visual result (---), Protein level (ug) (0), and Threshold (1).
- Printed Ticket Destination:** Includes fields for Book #, Sheet #, Cycle Number, Program Number, and Surface.
- Washing Process Information:** Includes dropdown menus for Identification, Washer brand, and Machine serial.

At the bottom right of the form, there are two buttons: a blue 'Save result' button and a blue 'bin' button. The status bar at the bottom indicates 'Incubator online (MINIPRO) 4444 0011' and 'Last readout: 05/02/2019 11:19 T: 58.9°C S: 2/60 Readings: 4/4'.

# PRO Tab

## Overlapping results

In case you start a new PRO analysis without having saved the previous results, the system will immediately inform this overlap through an emerging pop-up alert and a notification in red letters below the form. In this situation you must eliminate or save the actual (finished) readout so as to allow the system to update the program with the new reading given by the incubator. Once you have performed either elimination or saving of the previous result, your PRO tab will automatically be updated with the new reading and display the regular options.

The screenshot displays the Bionova PRO software interface. At the top, there is a menu bar with 'Bionova View WLRK Help' and a toolbar. Below the menu, there are tabs for 'Biological Indicators', 'Results', 'PRO', and 'PRO Results'. A table shows a list of test results:

#	Date	Brand	Product code	Batch	Monitoring Type	Result	Visual result	Protein level (µg)	Avg. Temp.	Book #	Sheet #	Cycle
1	05/02/2018 11:13:00	Chemdye	PRO1 micro		HTM01-01	Cancelled	---	0	38.9			
2		Chemdye	PRO1 micro		HTM01-01	---	---	0	0			
3		Chemdye	PRO1 micro		HTM01-01	---	---	0	0			

Below the table, there are sections for 'Incubation Data' and 'Incubator Performance Data'. The 'Incubation Data' section shows the date as 05/02/2018 11:13:00 and a remaining time of 09:39. The 'Incubator Performance Data' section shows an average temperature of 38.9. There is also a 'Protein pen data' section with dropdown menus for Product code, Brand, Batch, Monitoring Type, Result, Visual result, Protein level (µg), and Threshold. Below that is the 'Printed Ticket Destination' section with fields for Book #, Sheet #, Cycle Number, Program Number, and Surface. The 'Washing Process Information' section includes Identification, Washer brand, and Machine serial. At the bottom, a red warning message states: 'There is a PRO test in progress but the fields on this screen will not be updated until you save or discard the results from the previous test'. The status bar at the very bottom shows 'Incubator online (MR0PRO) 4444 0011' and 'Last readout: 05/02/2018 11:14 T: 39,9°C S: 2/90 Readings: 6/8'.

## IQAS

Monitoring reprocessing of instruments by the use of statistical control charts provides assurance that the process is working as anticipated, and the residual protein on instruments is within the expected range. An Internal Quality Assurance Scheme (IQAS) can be established with the aim of monitoring, over time, the efficiency of the cleaning process.

## Baseline development

The first step for an SSD is to measure reprocessed surgical instruments representing the full range of their workload to provide the basis upon which a monitoring system can be developed (Baseline).

There are two main approaches to instrument sampling:


- If a single measurement is to be made at each time point
- For measurements that fall naturally into groups established by the user (for example, five instruments per week)

In case of the single approach, a total of 20 measure-

ments should suffice initially from which parameters of the process can be estimated. In case of group more than 30 measurements will be required.

### • Procedure

- 1.1. Process your instrument as usual in your washer machine following the recommendations of the instrument manufacturer and the washer manufacturer.
- 1.2. Select a representative instrument for testing residual protein
- 1.3. Test the instrument selected with the Chemdye® PRO1MICRO device following strictly the instructions for use of the device.
- 1.4. Incubate the PRO1MICRO used according the device Instructions for use or the Autoreader User manual.
- 1.5. Before or during the incubation connect your autoreader to the Bionova® Traceability Software (version 3.2 or higher) following the autoreader User Manual.
- 1.6. Once within the Traceability Software, select the corresponding autoreader in the drop-down menu: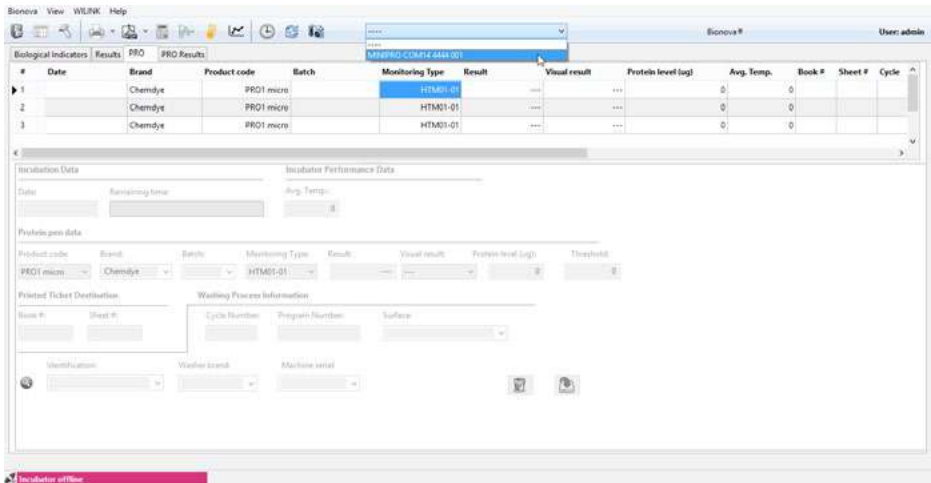


# PRO Tab


1.7. Select the incubation position where the Chemdye® PRO1MICRO is being incubated.

1.8. The first information displayed in the table refers to the Incubation Data and it will be filled automatically when the incubation finishes: Date and Average Temperature during the incubation.


During incubation: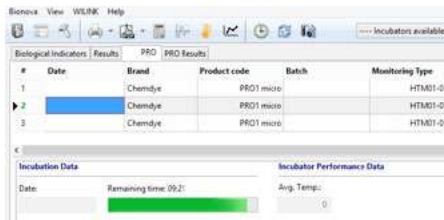

After incubation:

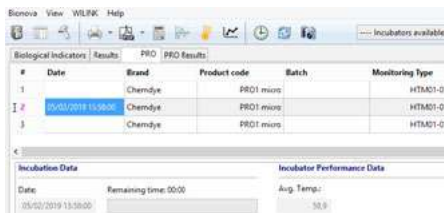1.9. The second information displayed in the table refers to the Protein Pen Data. Select PRO1MICRO as the product code (this field includes the option for saving the PRO1VT results) complete the product brand: Chemdye and batch: informed in the PRO1MICRO label. Monitoring type and results are filled automatically as well as Protein level and Threshold. Visual Result instead should be filled according to the color of the reactive solution after incubation.

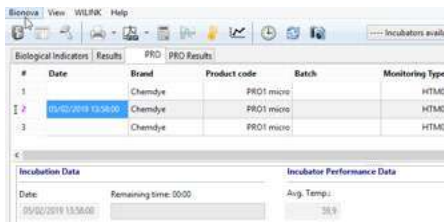
**CAUTION:** In order to record the visual result, remove the Protein Pen and verify the solution color at the moment the quantitative readout finishes.

1.10. The third information displayed in the table refers to the Printer Ticket Destination. Fill these fields with logbook number and page where the ticket will be saved. This is not mandatory information to fill.

1.11. The fourth information displayed in the table refers to the Washing Process Information. Add the main information of your washer machine to the program pressing:

#	Identification	Brand	Machine serial
1	Washer1	Steelco	2233
2	Washer2	Belimed	123456
3	Washer3	Matachana	709
4	Washer1	Miele	5427

The washer database will open. You can add a new washer, edit the information about a specific washer or select the washer used for reprocessing the instrument tested.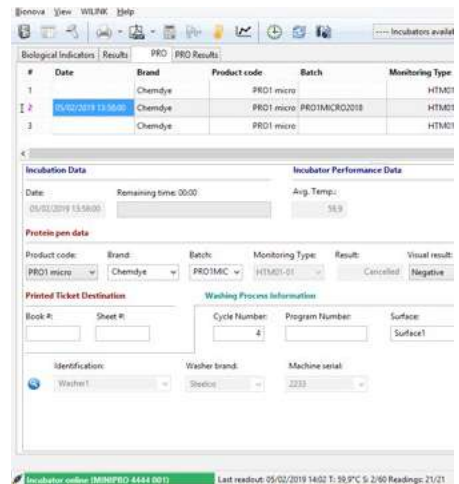

1.12. Once selected the washer machine, you must complete the information about the cycle number, program number and identify the surface/instrument tested.

# PRO Tab

1.13. Save the result pressing the button: 

1.14. You can delete all the information pressing the button: 

All results saved will be available in PRO Results tab. Repeat 1.1 - 1.13 steps anytime you want to record a test result of a washing instrument.


## Control Analysis

Ongoing Monitoring (HTM 01 - 01 Recommendation): A chart for each wash chamber/WD should be set up. The number and types of instruments should be tested as follows:

50 instruments per wash chamber/WD, at least every three months, chosen from difficult-to-clean instruments (for example, box joints, serrations, hinges, graters and reamers and complex retractors) where used. Other difficult-to-clean instruments should be identified and included in this testing.


### • Procedure

2.1. Once the first 20 measurements have been obtained, you can start the control analysis by pressing the button:  The graph tab will be displayed: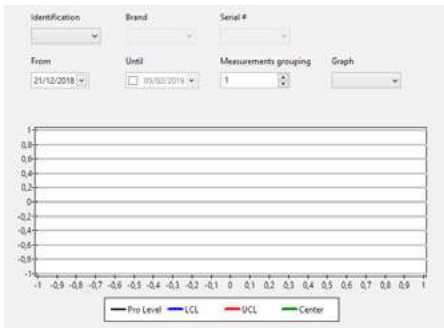

2.2. Select your washer by pressing the Identification drop-down menu. The Brand and Serial number of the washer will be filled automatically.

2.3. By default, the initial date of the analysis corresponds to the initial date of the first test result recorded. You can modify this information by pressing the corresponding drop-down menu.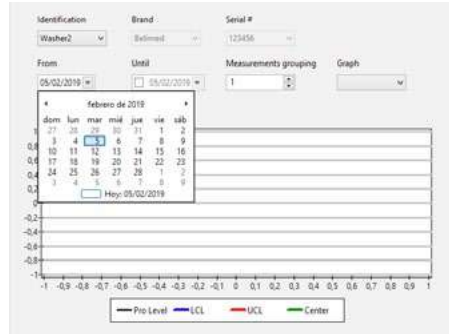


2.4. By default, the final date corresponds with the actual date. You can modify this information by pressing the corresponding drop-down menu.

2.5. Modify the measurement grouping option if you want to execute a group approach. You must consider more than 30 measurements for the baseline and 10 as a maximum number of measurements per group, in accordance with HTM 01-01.

2.6. If you want to continue with the single approach (measurement grouping = 1) two options will be displayed in the Graph Menu:

- I-Chart (Individual Chart): log<sub>10</sub> residual protein vs. measurement number.
- MR-Chart (Moving Range Chart): absolute difference between log<sub>10</sub> residual proteins vs. difference number.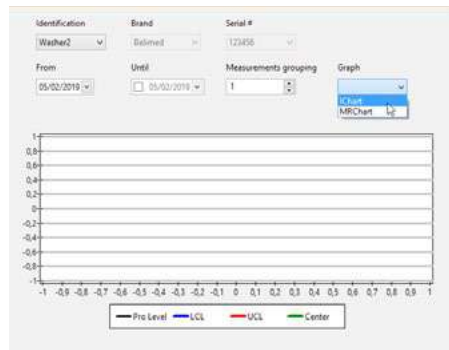


2.7. If you want to execute a group approach (measurement grouping >01) two options will be displayed

# PRO Tab

in the Graph Menu:

- XBAR-Chart: group average vs. group number.
- R-Chart (Range Chart): group range vs. group range number.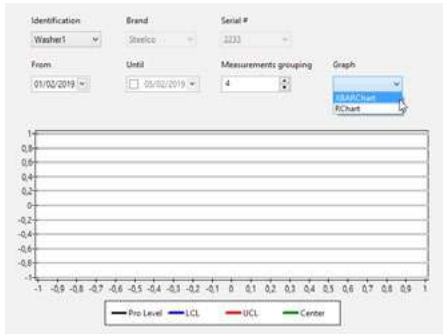

## Results interpretation

The monitoring of each process must vary randomly among the average and within the control limits (Upper control Limit and Lower Control Limit), and the process variation must be random and within below the control limit also. If a measure goes above the Upper Control Limit, the sample must be re-washed and the instruments must be sampled again. The maximum upper limit that is permitted in the standard is 5  $\mu\text{g}$  per instrument side (according to HTM 01-01). Anyway, measurements that fall outside of the control limits are an indication that the process is no longer in control. This may warrant an investigation to detect and eliminate any underlying causes.


Before interpreting the I-Chart (single approach) or XBAR-Chart (group approach), the MR-Chart or R-Chart (respectively) should be examined to determine if process variations are in control. If the MR-Chart / R-Chart are not in control, then the control limits in the I-Chart / XBAR-Chart cannot be considered approximate. Even when the measurements in the I-Chart / XBAR-Chart are within the control limits, it does not necessarily mean that the system is under control.

What is important is whether there appears to be any systematic behavior in sequential measurements. For example, if there were ten successive measurements all above the center line, then this might indicate that

a systematic change to the process has occurred, since we would expect half of these measurements to fall either side of the center line. To aid interpretation of statistical process control charts, a series of situations that may indicate an “out of control” process have been suggested (HTM01-01). Our software is prepared to monitor “Out of control” Schemes detecting this situations shown below.


**Example 1.** Out of control type 1: Instability.

In this case 8 or more consecutive measurements are on the same side of the average.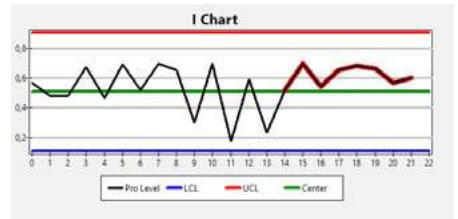


**Example 2.** Out of control type 2: Alternated.

In this case, there are 14 measurements where there is an alternating pattern.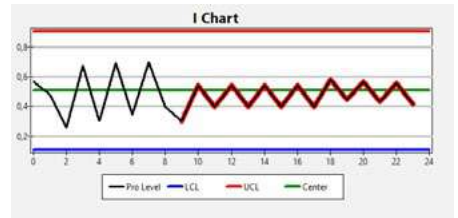

**Example 3.** Out of control type 3: Monotonic.

In this case 6 consecutive measurements present a monotonic trend (all increasing or decreasing).

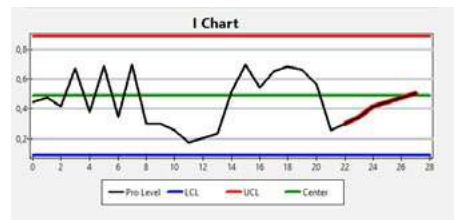## PRO Results Tab

This tab shows all saved results for Hygiene Monitoring tests performed with the incubator.

As in the BI results tab, this tab displays the same functionality and allows filtering and sorting of the PRO tests results. For more details, please read the *Sorting and Filtering of results* section.

Additionally, this tab possesses a column with a checkable field for every record. If it is checked, that measurement will be taken into account for the HTM01-01 calculations, otherwise it will be omitted.

The screenshot shows the Bionova software interface for the PRO Results tab. The window title is "Bionova" and the user is "admin". The interface includes a menu bar, a toolbar, and a search/filter area. The main area displays a table of test results. The table has the following columns: #, Lot, Serial #, Operator, Finish Date, Brand, Product code, Batch, Monitoring Type, Result, Visual result, Protein level, Avg. T, Book #, and Sheet #. The table contains 14 rows of data. The status bar at the bottom shows "Incubator online (IC1020FR0001 0001)", "Program 1: 1h / 60°C", "Program 2: 1h / 60°C", and "Last readout: 26/01/2018 11:15 T: 60.1°C S: 2/60 Readings: 33/3".

#	Lot	Serial #	Operator	Finish Date	Brand	Product code	Batch	Monitoring Type	Result	Visual result	Protein level	Avg. T	Book #	Sheet #
1	MN2PRO9999	1	admin	10/01/2018 08:38:09	Chembye	PRO1 micro	1	Quantitative	Negative	Positive	0	60.0	1	1
1	IC1020FR5017	502	admin	10/01/2018 08:38:05	Chembye	PRO1 micro	2	IQAS	Negative	Positive	0	59.9	-2	2
1	IC1020FRLCD077	2	admin	12/01/2018 15:35:55	Chembye	PRO1 micro	2	Quantitative	Positive	Positive	1.5	0.0	2	2
1	IC1020FRLCD025	3	admin	15/01/2018 09:53:38	Chembye	PRO1 micro	1	Quantitative	Positive	Positive	1	0.0	2	3
2	MN2PRO9999	1	admin	15/01/2018 11:17:36	Chembye	PRO1 micro	2	Quantitative	Negative	Positive	0	59.9	2	3
1	MN2PRO9999	1	admin	15/01/2018 11:17:33	Chembye	PRO1 micro	1	Quantitative	Positive	Positive	>10	59.9	1	1
3	MN2PRO9999	1	admin	15/01/2018 11:17:37	Chembye	PRO1 micro	1	Quantitative	Negative	Positive	0	59.9	1	1
1	IC1020FRLCD077	2	admin	17/01/2018 15:54:54	Chembye	PRO1 micro	222	Quantitative	Cancelled	Positive	0	0.0	2	2
1	IC1020FRLCD067	458	admin	25/01/2018 15:52:15	Chembye	PRO1 micro	2	IQAS	Positive	Positive	5.2	0.0	2	2
1	IC1020FRLCD077	2	admin	25/01/2018 15:52:20	Chembye	PRO1 micro	7	Quantitative	Positive	Negative	1,1	0.0	-7	7

## Equipment calibration

Before starting the calibration process, make sure that you won't need the incubator for the next 2-3 hours, because that's approximately what this process takes.

The temperature calibration tool available in Bionova® Software (2.1 Beta2 version or higher) allows adjustment of Bionova® incubator temperature. It is an easy-to-use tool that facilitates regular temperature calibration according to a procedure determined by the program. As a consequence of equipment's aging, some changes in the heating system happen. In order to compensate those small changes, the function allows a precise adjustment of the incubation temperature.

This new tool will enable to generate a calibration certificate for the equipment. Some of the info registered in this certificate is the institution to which the equipment belongs, staff who performed the procedure, equipment information, the reference device ID, environmental conditions and data on calibration.

The calibration process failure or operating errors could cause equipment malfunction. Make sure you have the knowledge and the proper equipment to perform this procedure safely. Terragene S.A. shall not be liable for any loss or damage that may result from the unsuitable use of the equipment or failures of the calibration tool.

### Calibration process

To perform this procedure, make sure you have an external thermometer, adequate for the thermometer hole available in the back of the incubator. Our TB-IC1020 digital thermometer has been developed with the appropriate specifications. In case you use a different brand, make sure the specifications match with those of TB-IC1020.

To start calibration process, press  button and follow the program's instructions.

Only start calibration process when the equipment temperature is stable.

After calibration has finished, a certificate of calibration will automatically appear on your screen. You will be able to print or save it.

## Other functionalities

### Clock synchronization

By pressing the icon  you can synchronize the incubator's clock (date and time) with that of your computer's system.

### Full screen mode

The program offers a "Full Screen Mode". This mode can be activated by pressing the F11 key or through corresponding option in the menu.

View >> Full Screen Mode

### Screen captures

The application provides a function that allows taking a screenshot and saving it in a folder at the user's choice. This function can be accessed through the  button on the toolbar. This feature serves the user to easily take a snapshot in the event that the program is defective, so that it can then be sent for analysis.

### Software and incubator information

The application includes a screen through which various data can be consulted, such as the serial number of the incubator and the program version.

This screen is accessed through the option:

Help >> About ...

### WILINK wizard

The program offers a tool assistant for configuring our WILINK accessory, which can be used for making the Ethernet incubators compatible with Wi-Fi networks. This tool can be opened through the option:

WILINK >> Wmlink Wizard

### Software requirements

The application is compatible with Windows 7, 8 and 10 (32/64 bits versions). In terms of software, the application does not raise any additional requirement, for the installer already contains all the necessary elements for its operation.

### Hardware requirements

The program is extremely light in relation to the use of the system resources, so it should work without problems in any computer model that has come to market in the last four years.

As a general guideline, these are the minimum recommended hardware requirements:

- Microprocessor: Intel Atom 330
- RAM: 2 GB
- Display: 1360 x 768 resolutions
- Other:
  - Free USB ports to allow the connection to incubators.
  - Ethernet port to connect and configure WILINK.
  - Network connection to access Ethernet incubators (no internet required).

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# Index

<b>Introduction</b> .....	<b>3</b>
Incubator connection .....	3
First execution .....	3
Starting the application .....	4
Access rights and users commutation.....	4
User account management .....	4
Password reset and modification .....	4
<b>Language configuration</b> .....	<b>5</b>
Language selection .....	5
How to force the language selection .....	5
<b>Main screen</b> .....	<b>6</b>
Status bar .....	6
Incubator selection.....	6
<b>Biological Indicators Tab</b> .....	<b>7</b>
Color codes.....	8
Expedited data upload.....	8
Biological Indicators barcode reader .....	9
Form view.....	10
BI's results storage.....	11
Automatic storage .....	11
Manual storage .....	11
<b>Results Tab</b> .....	<b>12</b>
Sorting and filtering of results.....	12
Sterilization Process Risk (SPR) calculation .....	13
Historical Sterilizer Performance.....	14
<b>PRO Tab</b> .....	<b>18</b>
Storing PRO Results .....	19
Overlapping results .....	20
IQAS.....	21
Baseline development.....	21
Control Analysis .....	23
Results interpretation .....	24
<b>PRO Results Tab</b> .....	<b>25</b>
<b>Reports generation</b> .....	<b>26</b>
Printed reports .....	26
Generating reports in PDF format.....	26
<b>Equipment calibration</b> .....	<b>27</b>
Calibration process .....	27
<b>Other functionalities</b> .....	<b>28</b>
Clock synchronization .....	28
Full screen mode .....	28
Screen captures.....	28
Software and incubator information .....	28
WILINK wizard.....	28
Software requirements.....	28
Hardware requirements.....	28
<b>Index</b> .....	<b>58</b>

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