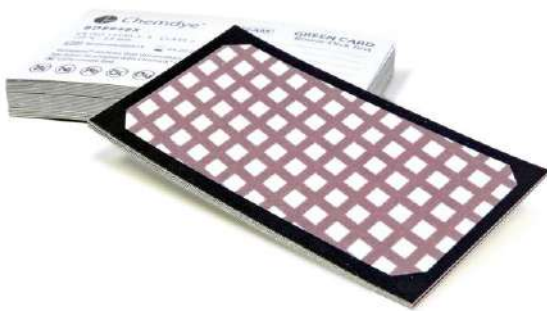
BD8948X

Indicador Químico

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Uso previsto

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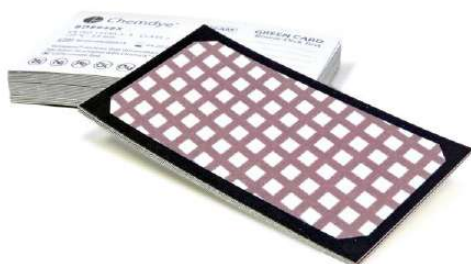
Bowie & Dick Test Card

Usage

Bowie & Dick test monitors the effectiveness of air removal in vacuum-assisted steam sterilizers. Designed for cycles of 3.5 minutes at 134 °C // 4.0 minutes at 132 °C.

Applicable Regulation

ISO 11140-1:2014 and ISO 11140-4:2007

Product Classification

Class 1, according to risk.

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012.

Characteristics

Type 2 Chemical Indicator

Initial Color: purple

Final Color: green

Size: 89 mm x 48 mm x 0.33 mm

Material: 240 g/m² one-side-laminated cardboard.

Weight: 1.2 g.

Cycle information can be recorded on the back of the card.

NOTE:

Air removal problems are shown as heterogeneous color change, i.e. the presence of purple color.

100 % Toxic Heavy Metals free

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 10-30 °C, RH 30-80 %, store in dark place.

Transportation conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Shelf-life

5 years

Packing

120 units per box.

Packing information: product code and description, presentation, process for intended use, classification according to regulation, directions for use, manufacturer information and data printed on the product.

Labelling

Printed on product: Product code and description, process for intended use, classification according to regulation, manufacturer information, batch number and expiration date.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical Industries.

Other important information (optional)

Read product's directions for use thoroughly before use.

Note: When necessary, limits for each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization process other than Steam.

Do not use a Bowie & Dick Test Card in gravity displacement steam sterilization cycles (where there is no pre-vacuum phase) or any kind of sterilization processes other than vacuum-assisted Steam.

Do not reuse the sterilizer until the card has changed homogeneously into the color indicated in the product.