Infection Control Division | **Chemical Indicators**

CT10 Indicator Tape

For Ethylene Oxide sterilization processes

Usage

For monitoring Ethylene Oxide (EO) sterilization processes.
External use.

Conditions: 600 mg/l EO, 60 % RH, 54 °C, 20 minutes // 600 mg/l EO, 60 % RH, 37 °C, 25 minutes.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.
ISO 11140-1:2014.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk.

Characteristics

Type 1 Chemical Indicator.
Roll of self-adhesive tape printed with indicator ink.
Size: 50 m x 18 mm.
Initial color: **orange**.
Final color: **green**.

100 % Toxic Heavy Metal free.

Tape surface treated with covering materials that allow an easy and continuous writing. Adhesive capable of adhering to wrap cloth, paper and plastic. Designed with advanced technologies to ensure that the tape remains adhered to the surface after sterilization and leaves no trace of adhesive on the surface of materials after removal.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light.

Transport conditions

Storage conditions should be strictly followed.
Transport in closed and reinforced boxes in order to avoid damages.
The transport of this product does not represent a risk for health.

Shelf life

5 years.

Infection Control Division | **Chemical Indicators**

CT10 Indicator Tape

For Ethylene Oxide sterilization processes

**Packing**

1 roll of 50 m x 18 mm.

Packing information: product description, classification according to regulation, storage conditions, manufacturer information and data on packing's label.

Labeling

On roll's cone: process for intended use, classification according to regulation, batch number, expiration date and manufacturer.

On product's packing: product code and description, process for intended use, use conditions, classification according to regulation, color change, tape dimensions, batch number, manufacture and expiration date, bar code and datamatrix code.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical Industries.

Other relevant information

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to Steam, Dry Heat or any sterilization processes other than EO.

Do not reuse the sterilizer until the indicator has changed into the color indicated in the product.

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