# KBD8948X

## Bowie-Dick Test Card Kit

*Includes Bowie & Dick Test Card Chemical Indicator and its Holder.*

### Usage

Bowie-Dick test monitors the effectiveness of air removal in vacuum-assisted steam sterilizers. Designed for cycles of 3.5 minutes at 134 °C, 4.0 minutes at 132 °C.

### Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.  
ISO 11140-1:2014 and ISO 11140-4:2007

### Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

### Classification

Class 1, according to risk (ANMAT).

### Characteristics

#### BD8948X

Bowie-Dick Test Card Chemical Indicator.

Type 2 Chemical Indicator

Initial Color: **purple**

Final Color: **green**

Size: 89 mm x 48 mm x 0.33 mm

Material: one-side-laminated cardboard.

Cycle information can be recorded on the back of the card.

NOTE: Air removal problems are shown as heterogeneous color change, i.e. the presence of purple color.

100 % Toxic Heavy Metals free

#### BD8948H

Stainless steel re-usable Holder for Bowie-Dick Test Card of great resistance and durability coated by a protective layer.

Keeps BD8948X Test Card in place for proper assessment of sterilization cycle.

Height: 42.7 mm

Base: 133.2 mm x 72.4 mm

Upper part where the BD8948X Test Card fits: 93 mm x 53.5 mm

### Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

### Storage conditions

T = 10-30 °C, RH = 30-80 %, store in dark place.

**Transport conditions**

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

**Shelf life**

5 years

**Packing**

1 Holder +120 Bowie-Dick Test Cards per box.

Packing information: product description, process for intended use, regulation, presentation, storage conditions, manufacturer information and data printed on the product.

**Labelling**

On products' box: Product code and description, process for intended use, classification according to regulation, manufacturer information, batch number, expiration date, barcode and datamatrix code.

**Possible target markets**

Healthcare, Food, Pharmaceutical and Medical Industries.

**Other important information**

Read product's instructions for use thoroughly before use.

**Precautions**

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization process other than Steam.

Do not use a Bowie-Dick Test Card in gravity displacement steam sterilization cycles (where there is no pre-vacuum phase) or any kind of sterilization processes other than vacuum-assisted Steam.

Do not reuse the sterilizer until the card has changed homogeneously into the color indicated in the product.

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